


TREATING A PROLAPSE  
OR  
LIGATING ANY LARGE TISSUE MASS

with  
**The Callicrate Bander  
& VA Band**


# You will need: The Callicrate Bander and a Callicrate VA Band

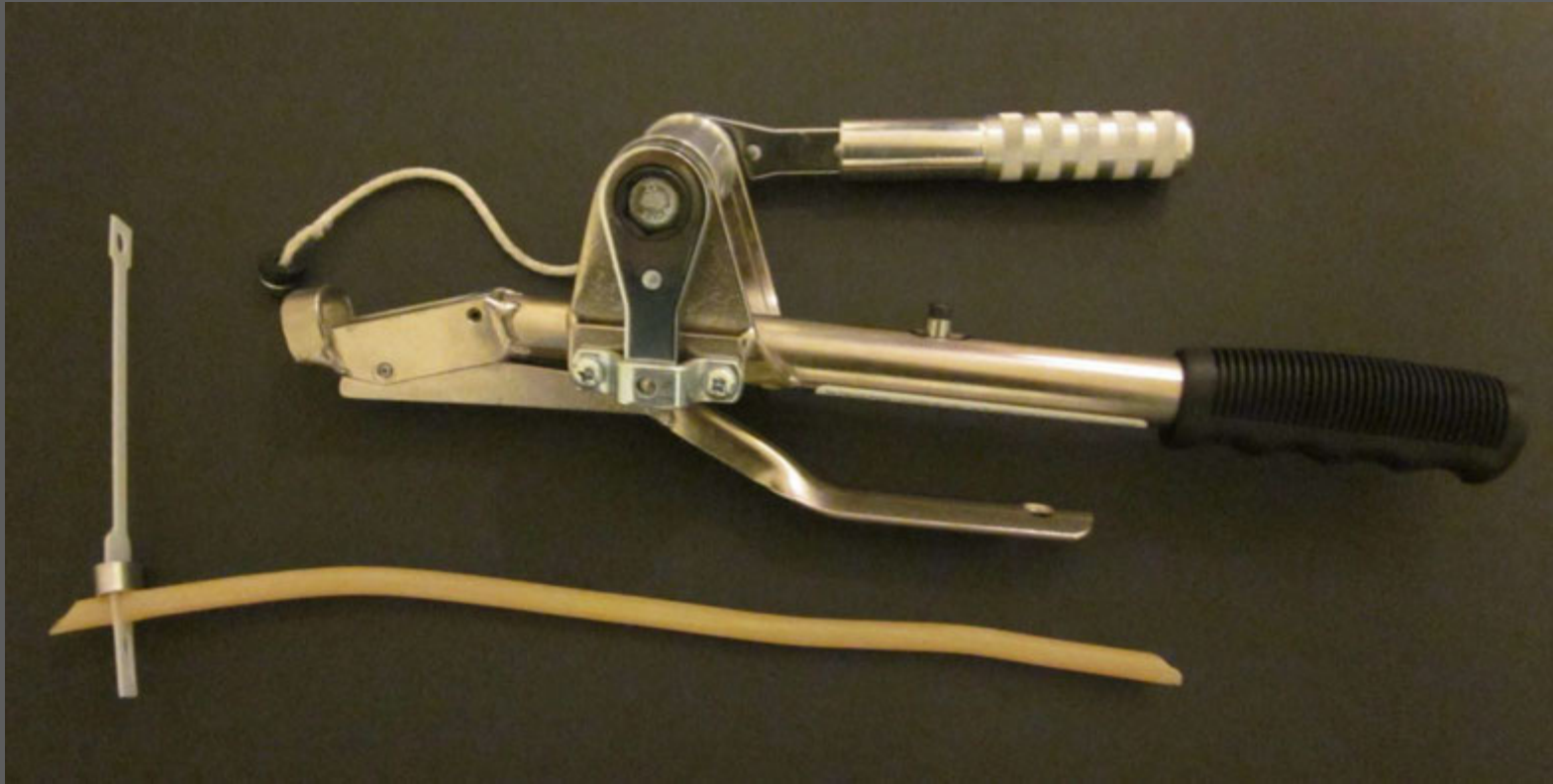


Tetanus toxoid (not antitoxin) must be used. It is important to read and follow vaccine instructions. For more information, contact your veterinarian.

WEAR SAFETY GLASSES

Move the button on the string  
close to the spool on the bander

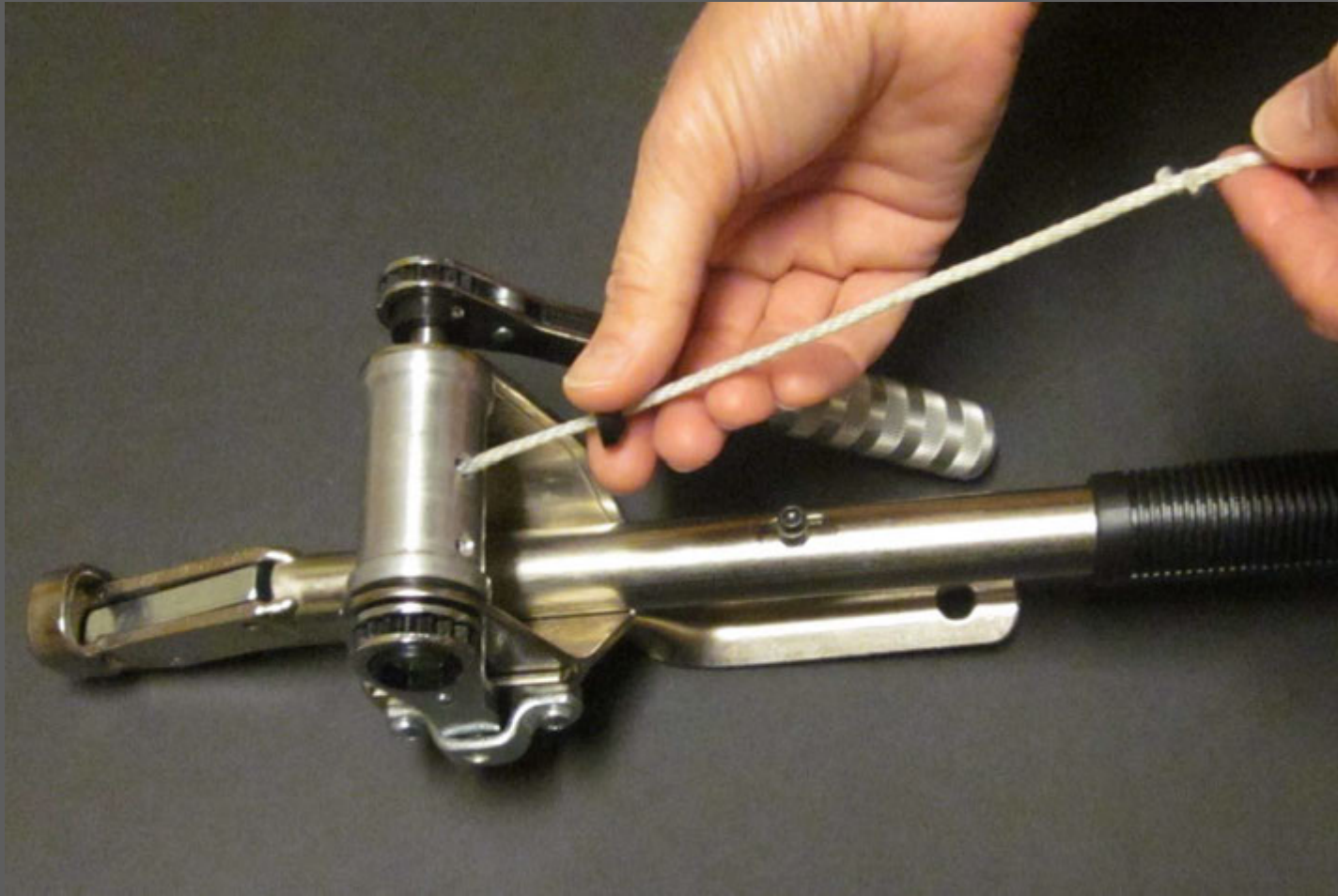


Place the VA Band in the nose of the bander with the clip seam side up

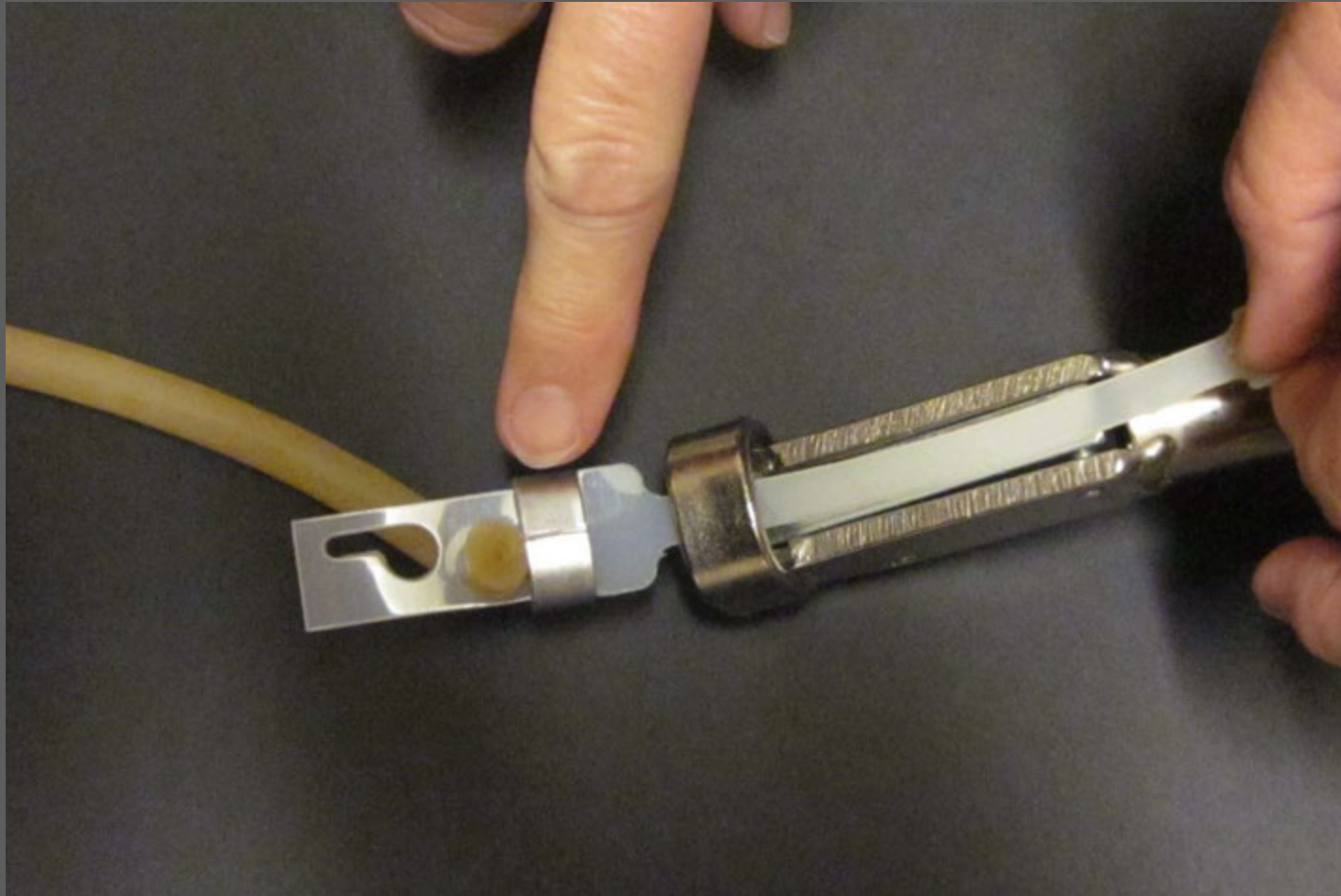




Place the knot on the end of the string through the hole on the end of the tab

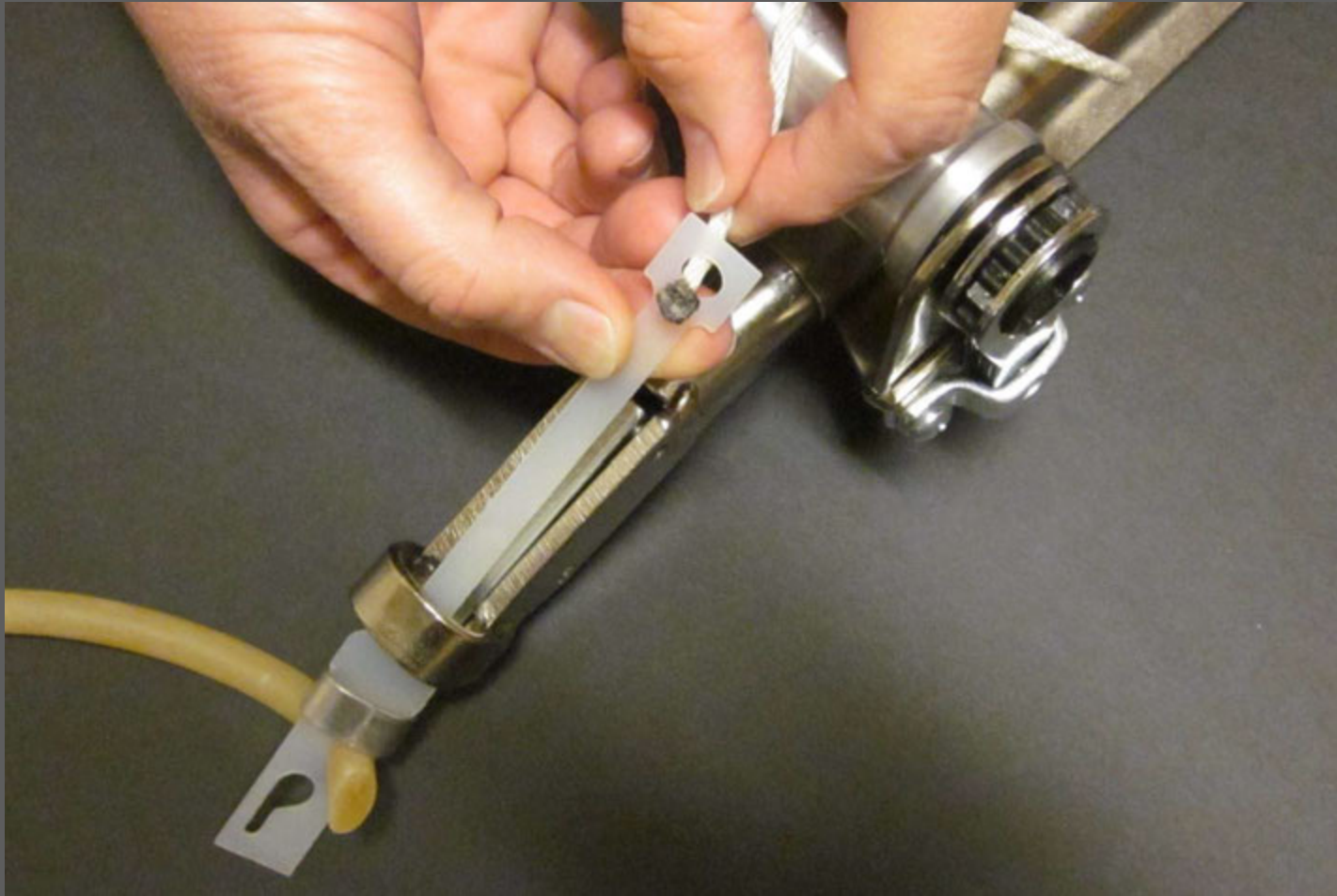

Position the button, washer, and plastic tab close to the knot end of the string

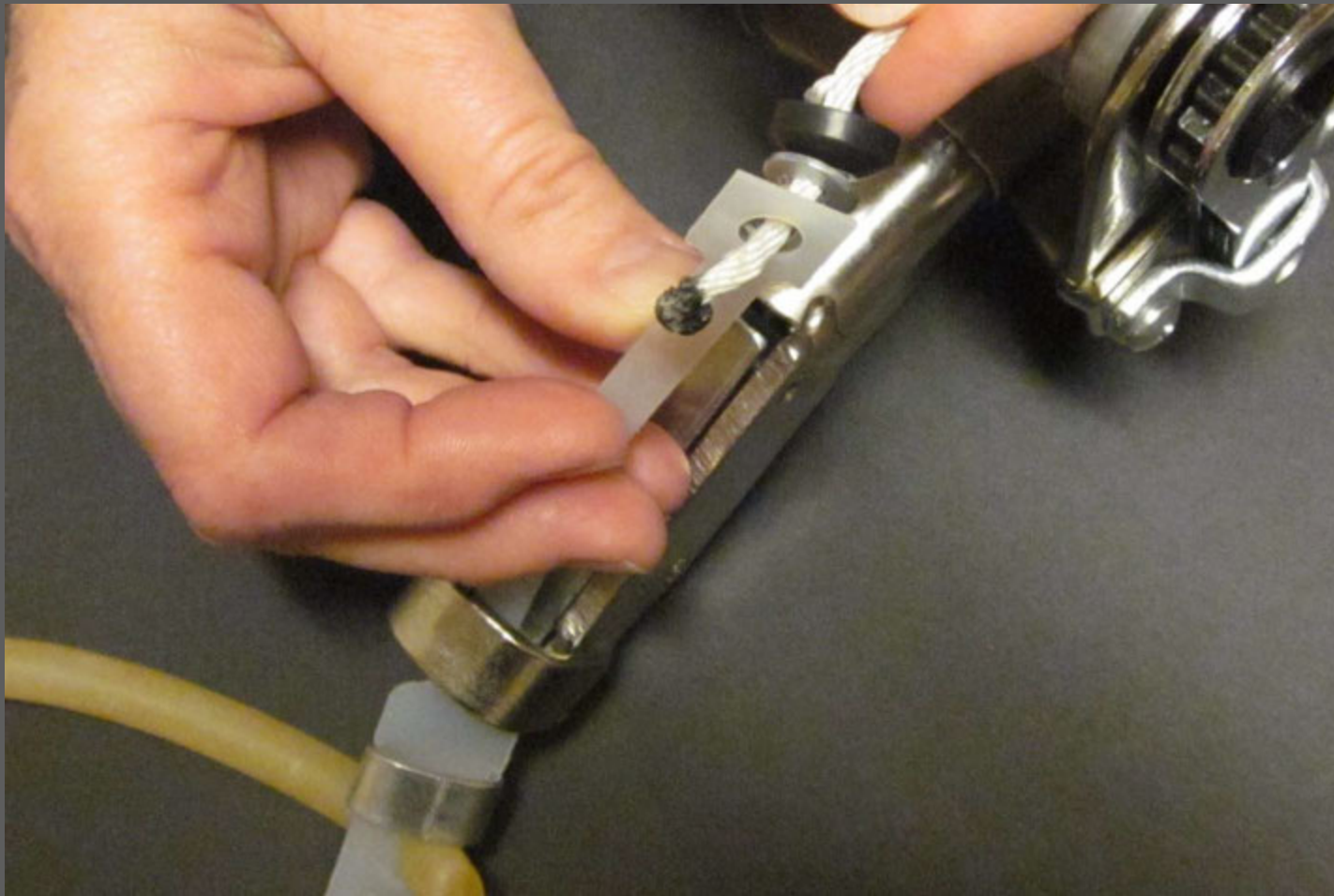


Tie a single overhand knot in the string -  
include the button in the knot





Ratchet until the slack is out of the string and the tab

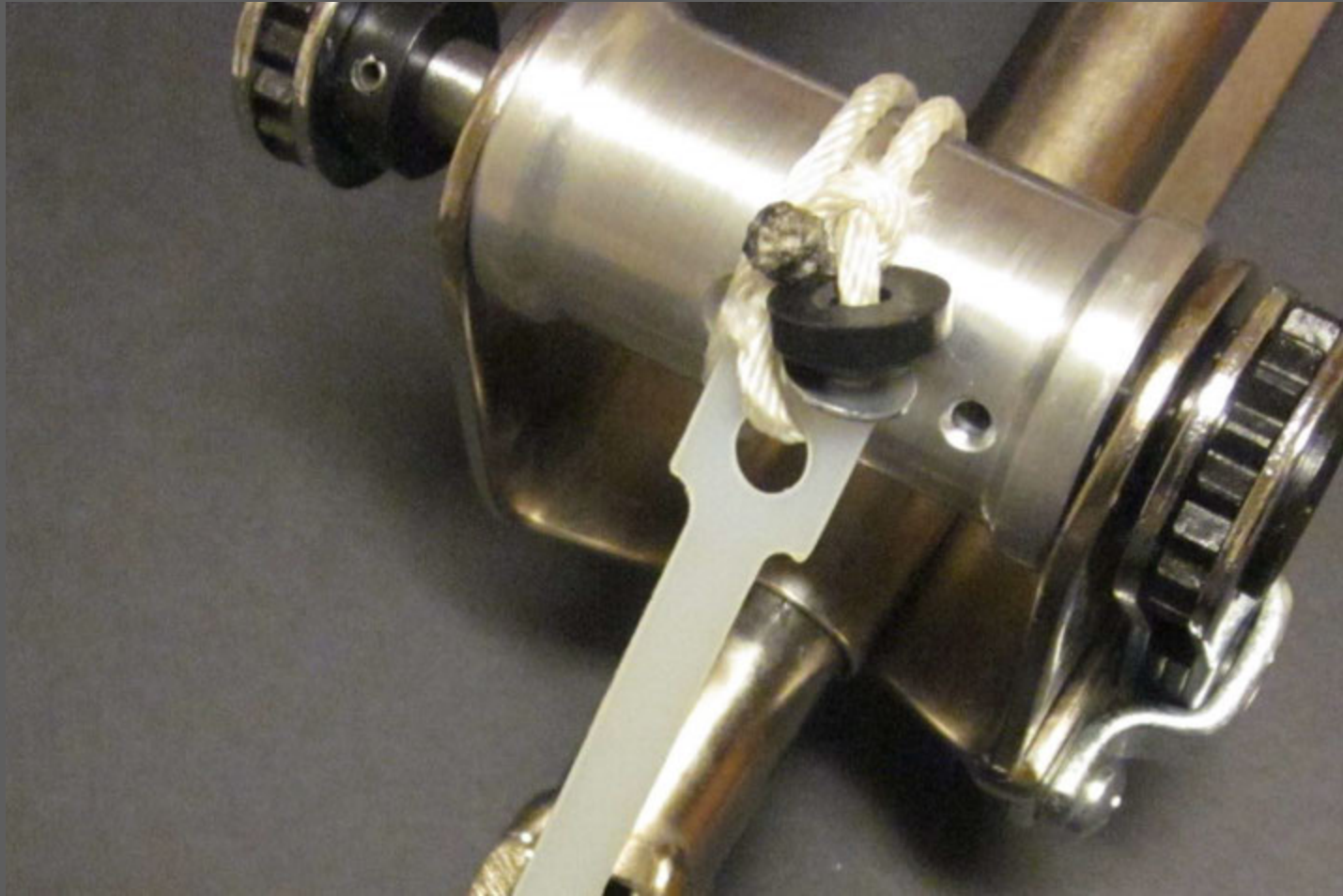





Wrap the VA Band around the mass you plan to ligate



Place the loose end of the band into the slotted hole at the end of the tab

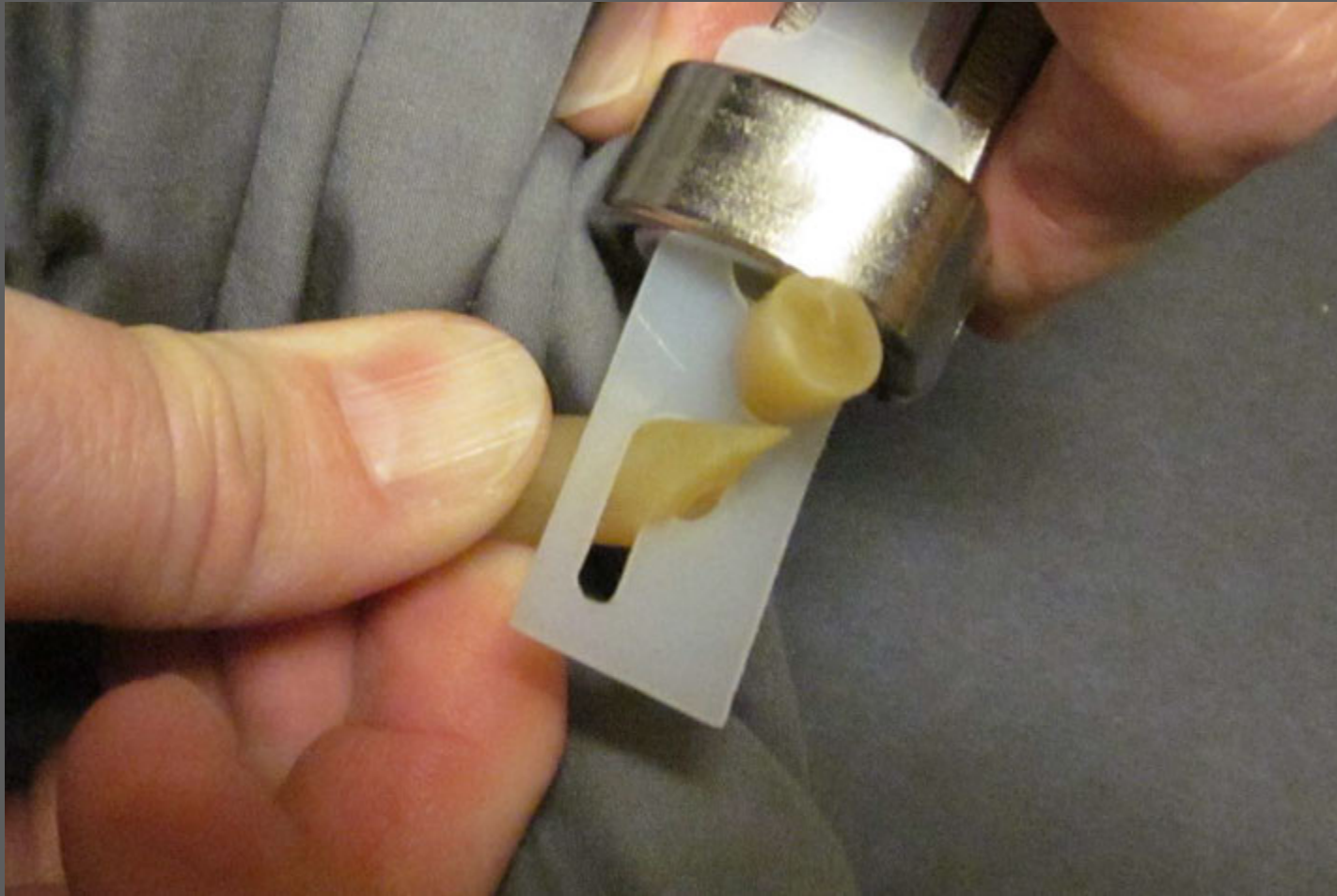

Pull VA band through slot to  
take up slack

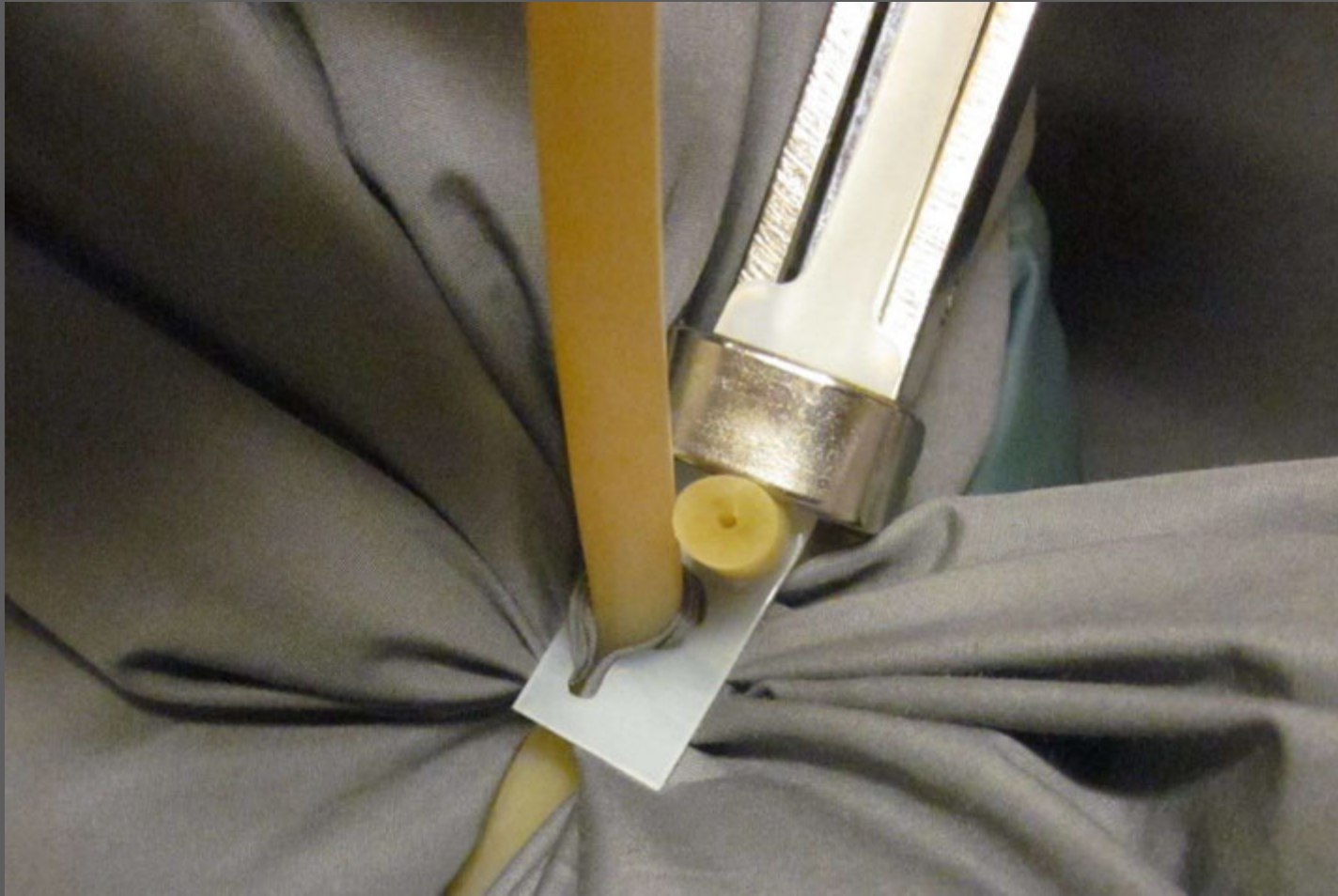



Use Callicrate Cutter to cut  
off excess tubing





Place a drop of liquid soap on each end of the latex band




If you do not lubricate, the plastic tab may break before the ends of the band pull through the clip.

Ratchet, making sure the band wraps evenly on each side of the spool



Tighten until the tension indicator  
peg touches the back of the slot

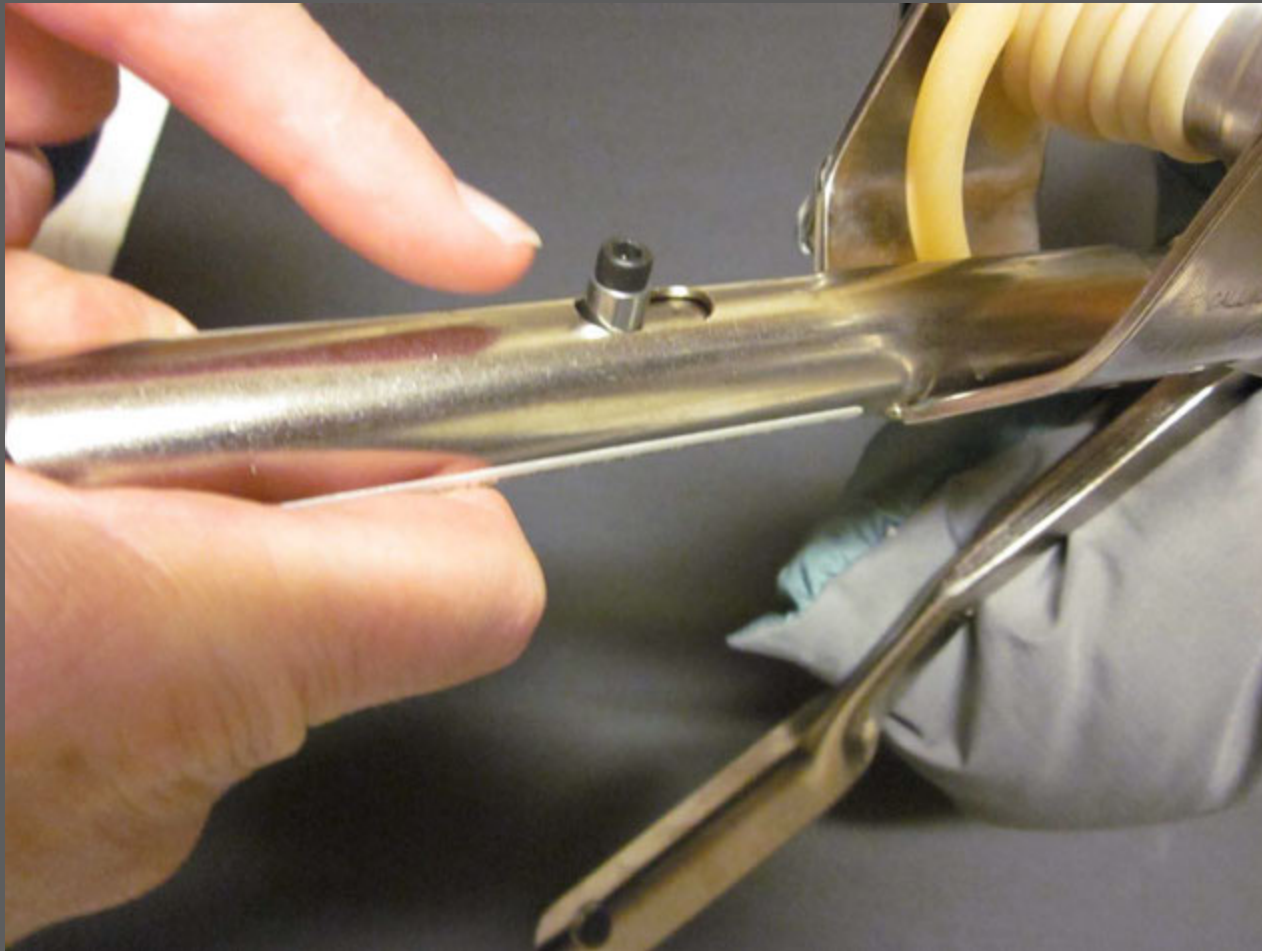



Crimp the clip by pushing down on the crimping handle



Using the Callicrate Cutter, cut the band as close to the spool as possible



The prolapse may need to be surgically removed after banding depending on the size of tissue mass.  
Contact your veterinarian if assistance is required.