

**it**  
**ORTHOPEDIC**  
**SOFT SUPPORTS**

**Safe.**

- **Miracle Knee™ Support**
- **Knee Compression Strap**
- **Dual Strap Ankle Support**

**Comfort, Support And Mobility**

Professional's Choice Orthopedic Soft Supports provide complete mobility at all levels of activity. Our designs allow for a custom fit so you can play sports, work, dance, walk and travel without pain.



**Professional's Choice Sports Medicine Products, Inc.**

# The Only Thing You Stand To Lose Is Pain

Stay active with our high-quality Orthopedic Soft Supports. Technologically advanced materials provide uniform compression and support for strained ligaments, tendons and muscles. And, the limestone based neoprene in the Miracle Knee Support and Knee Compression Strap holds in body heat for moist heat therapy which relaxes the muscles and increases circulation.

Reduce the effects of arthritis and damaged cartilage with our **Miracle Knee Support**. Plus, the wrap design and open back provide a perfect fit from thigh to calf. No more pinching, binding or rubbing. From simple support and compression for preventing injuries to post-arthroscopic surgery rehabilitation – the Miracle Knee is your best choice.

Our **Knee Compression Strap** offers relief from acute and chronic pain associated with Jumper's Knee, patella tendinitis, Osgood-Schlatter disease and Chondromalacia. The innovative fastening system allows for controlled compression to the knee area without compromising circulation or comfort.

The **Dual Strap Ankle Support** provides compression and support to both the ankle and arch. Perfect for acute or chronic ankle problems. The sock design fits into most shoe styles and provides adjustable support for ankle instability. Plus, the dual strapping system simulates athletic taping procedures.

**For more information on our other products call:  
(800) 331-9421**

## Miracle Knee Support Application Instructions




1. Position the patellar opening about 1" above the center of the knee cap.
2. Attach the thigh strap so the top is flush with the support. Pull the support down so the opening spreads around the knee cap.
3. Attach the calf strap to the support.
4. Adjust the straps for a comfortable fit.

## Knee Compression Strap Application Instructions



1. Stretch out and position the strap.
2. Wrap the end without hook material around the leg.
3. Attach the inner hook fastener directly to the lateral side of the support.
4. Attach the end hook fastener to the support, applying compression to the patellar tendon.
5. Adjust the strap for a comfortable fit.

## Dual Strap Ankle Support Application Instructions

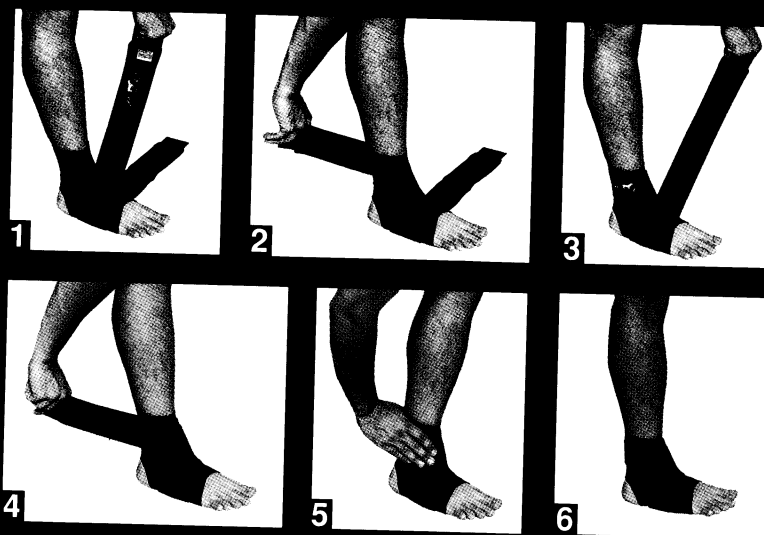

1. Firmly pull up the outside strap.
2. Wrap outside strap around inside of ankle. Attach hook fasteners.
3. After applying outside strap, firmly pull up on inside strap.
4. Wrap inside strap around outside of ankle. Attach hook fasteners.
5. Be sure both straps are properly secured.
6. Adjust straps for a comfortable fit.

## Washing Instructions

Hand wash in cool water with a mild detergent, such as Professional's Choice SportsSuds™. Rinse thoroughly to remove any soap residue, then towel off the excess moisture and allow to air dry.



**2709 Via Orange Way**  
**Spring Valley, CA 91978**  
**(619) 670-2300**  
**(800) 331-9421**  
**<http://www.pcsmp.com>**

A portion of the proceeds from your purchase will be donated to the North American Riding For The Handicapped Association and other worthwhile charities.

© 5/99 Professional's Choice