



Equine Leggings System

INSTRUCTIONS FOR USE - Please read carefully

Congratulations on your equine legging purchase.

General Step by Step Guidelines:

1. The horse should have its head tied up or placed in a set of cross ties. In some cases the use of a muzzle may be utilized.
2. All dirt, poultice, liniments or any topically applied substances should be removed prior to use.
3. Pre-warm the leggings by plugging them in and placing them on the high setting for a few minutes.
4. Place the leggings on the desired limbs (short pads on the top portion of the legs) and secure the Velcro straps. Patience must be exercised when doing this for the first time, as this is a strange concept for the horse to understand.
5. Keep all cords and appliance out of reach of the animals' mouth and feet.
6. Turn the switch on the low setting initially until the animal becomes acquainted with the therapy. After a few minutes, set the switch to high for 10-15 minutes.
7. Continue the therapy at the low setting for the duration of the treatment.
8. When the treatment is concluded, the inside of the leggings should be wiped down with a mild disinfectant, dried and stored in a clean place.
9. A rigorous massage to the tendons and ligaments after therapy is ideal.

Important Safety Instructions

- Never use the leggings along with any type of liniment, salve or ointment.
- Do not plug in and leave unattended. Check hide under device frequently.
- Do not immerse the leggings cover in water unless the electrical pads and cord have been removed.
- Do not use pins or other metallic objects to hold legging cover in place.
- Never use pads without outer cover in place.
- Periodically examine the inner cover (white PVC). Return device for repair if inner covering shows any sign of deterioration.
- Do not fold or bend the heat element.
- Do not pull the leggings by the cord or use the cord as a handle.

One Year Limited Warranty

WARRANTY: The Company warrants this product, to the original purchaser or gift recipient, to be free from defects in workmanship and materials under normal use and service, for a period of one year from the date of purchase.

LIMITATIONS: ALL WARRANTIES IMPLIED BY STATE OR PROVINCIAL LAW, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE EXPRESSLY LIMITED TO THE DURATION OF THE


LIMITED WARRANTIES SET FORTH ABOVE. Some jurisdictions do not allow limitation on how long an implied warranty lasts, so the above limitation may not apply to you. IN NO EVENT SHALL THE COMPANY BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES, LOSS OF PROFIT OR MEDICAL EXPENSES CAUSED BY ANY DEFECT, FAILURE, MISUSE, OR MALFUNCTION OF THE PRODUCT.

Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you. The Company will not be responsible for damages or losses, direct or indirect, caused by misuse, abuse, accident, negligence, conditions of transportation or storage, or failure to follow instruction. The Company will not be responsible for any statements that are made or published, written or oral, that are inconsistent with this written warranty, or which are misleading or inconsistent with the facts as published in the literature or specifications furnished by the Company. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Warranty Claim Procedure: Any product that proves defective within the warranty period will, transportation charges pre-paid, be repaired or replaced at the option of the Company, free of charge. Simply include your purchase receipt and deliver to:

Thermotex Therapy Systems Ltd.
#15 – 6115 – 4 Street, S.E.
Calgary, Alberta T2H 0H9
1-800- 975-0253
www.thermotex.com

Assembly Instructions

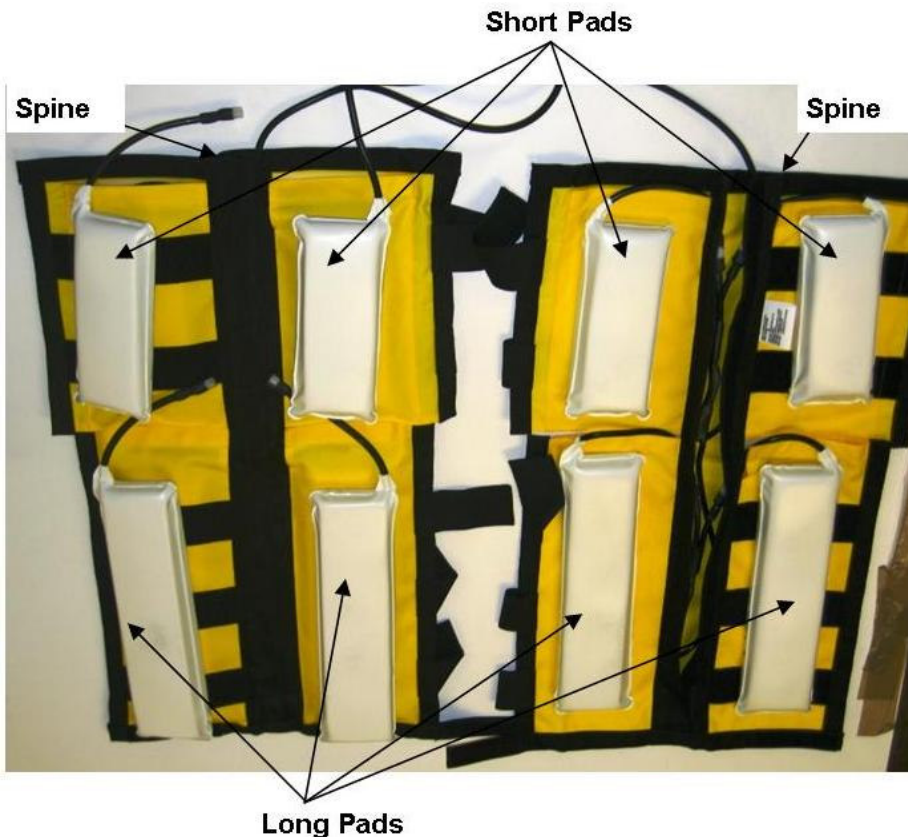

- * There are 8 pads in the legging set, as shown; 4 long and 4 short.
- * Each legging has 2 of each size.
- * The “hard side” of the pad should be placed inside the legging cover so that the inward side is touching the animal when the legging is in place.
- * Each pad plugs into the harness which runs up the spine of the legging.