

Equine TMJ Hood System

Congratulations on your equine TMJ Hood system purchase.

INSTRUCTIONS FOR USE - Please read carefully

General Step by Step Guidelines:

1. Secure the horse in cross ties.
2. Brush any dirt away from the head and loosen the halter one or two notches.
3. Pre-warm the TMJ Hood system by plugging in the appliance and placing it on the high setting for a few minutes before placing on the patient.
4. Fasten the TMJ Hood comfortably on the patient using the Velcro Straps.
5. Turn TMJ Hood appliance on the high for first 10 minutes of therapy.
6. After the initial ten minutes, turn the heat to the low setting and maintain this setting until the therapy session is complete.
7. Disinfect the inside of the TMJ Hood after use and allow it to dry before storage in a clean place.

Important Safety Instructions

- Never use the TMJ Hood along with any type of liniment, salve or ointment.
- Do not plug in and leave unattended. Check hide under device frequently.
- Do not immerse the TMJ Hood cover in water unless the electrical pads and cord have been removed.
- Do not use pins or other metallic objects to hold bed cover in place.
- Never use pads without outer cover in place.
- Periodically examine the inner cover (white PVC). Return device for repair if inner covering shows any sign of deterioration.
- Do not fold or bend the heat element.
- Do not pull the TMJ Hood by the cord or use the cord as a handle.

One Year Limited Warranty

WARRANTY: The Company warrants this product, to the original purchaser or gift recipient, to be free from defects in workmanship and materials under normal use and service, for a period of one year from the date of purchase.

LIMITATIONS: ALL WARRANTIES IMPLIED BY STATE OR PROVINCIAL LAW, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTIES SET FORTH ABOVE. Some jurisdictions do not allow limitation on how long an implied warranty lasts, so the above limitation may not apply to you. IN NO EVENT SHALL THE COMPANY BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES,


LOSS OF PROFIT OR MEDICAL EXPENSES CAUSED BY ANY DEFECT, FAILURE, MISUSE, OR MALFUNCTION OF THE PRODUCT.

Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you. The Company will not be responsible for damages or losses, direct or indirect, caused by misuse, abuse, accident, negligence, conditions of transportation or storage, or failure to follow instruction. The Company will not be responsible for any statements that are made or published, written or oral, that are inconsistent with this written warranty, or which are misleading or inconsistent with the facts as published in the literature or specifications furnished by the Company. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Warranty Claim Procedure: Any product that proves defective within the warranty period will, transportation charges pre-paid, be repaired or replaced at the option of the Company, free of charge. Simply include your purchase receipt and deliver to:

Thermotex Therapy Systems Ltd.
#15 – 6115 – 4 Street, S.E.
Calgary, Alberta T2H 0H9
1-800- 975-0253
www.thermotex.com

Assembly Instructions

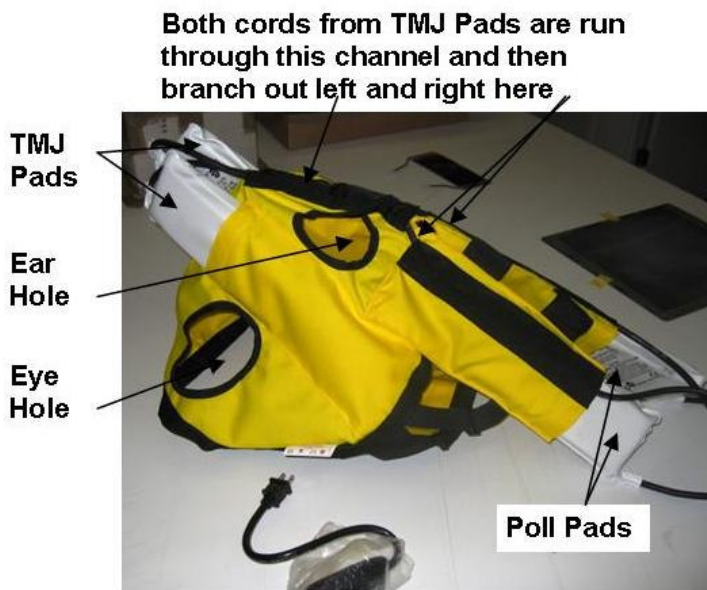

- * There are 4 pads in the TMJ Hood system
- * Two pads are positioned along side the TMJ (temporomandibular joint) and two pads along the poll.
- * The “hard side” of the pads should be placed inside the TMJ hood cover so that the inward side is touching the animal when the TMJ Hood is in place.
- * Each pad plugs into the harness which runs along a top spine that branches into two veins to accommodate the pads used for the treating the poll area