

VET-JET

POWER DEHORNING SAW

Model #350M

**Only a Saw
Dehorns Without
Crushing**



**Incorporates
All The Features
The Job Demands**

DESCRIPTION

PRACTICAL AND HUMANE

Sold throughout the United States and Canada for over 30 years The Vet-Jet has earned the respect of Veterinarians and Cattlemen, as being the most humane way to Dehorn Cattle. Animal torment and struggle held to a minimum. No bruising or crushing. Horn is usually off before the animal senses any pain.

DESIGN AND ADAPTABILITY

Operator can sever the horn at any desired location, from Tipping to Close Head Shaping. Opening the head for easy access to arteries. The Vet-Jet has a choice of two bottom guard Plates. A #4 which is standard and comes with the Unit has a 4" open end hole. Designed for most dehorning close and Head Shaping. A #3 that has a 3" dia. Closed end Design (for extra Ear Protection) on 1-1/2 dia. and under horns.

QUALITY

The Vet-Jet is built sturdy and will Warrant years of satisfactory Service.

SAFETY

The Vet-Jet has proven to be a very safe instrument over 30 years of Service.

SPEED AND EASE OF OPERATION

The Vet-Jet is a well balanced tool with two hand grips interchangeable for Right or Left Hand Operation. And again the cut is Clean and exposes the arteries for extraction.

OPERATING INSTRUCTIONS FOR MODEL #359A DEHORNER

Although this can vary in different situations (horn angle, breed, etc.);

- **place the inner arc of the rotating blade guards against the desired location where you wish to cut,
- **start the motor
- **push forward

Even though the rotating guards do provide reasonable ear protection, it is still advisable to restrain the head as to avoid a direct path with the ear. With a little practice, you will find the proper approach to fit your situation best.

CLEANING

- **Remove the bottom plate
- **Stand dehorner on end (motor up)
- **Flush blood, horn fragments and hair off with hose
***** (CAUTION: DO NOT GET WATER IN THE MOTOR) *****
- **Keep rotating guards free and clean
- **Dry blade
- **Coat with Vaseline or oil

*****WARRANTY*****

As with the Milwaukee Warranty on the motor, Fullenkamp Enterprises, Inc. also warrants the unit to be free of defects. Should you find that the unit will not operate as described, due to defective material or workmanship, the defective parts will be replaced with new parts at no charge.

If the unit needs to be returned to us for repair, due to defective parts, Fullenkamp Enterprises will reimburse the shipping cost (least expensive way). If the unit needs to be returned due to normal wear or is damaged by accident or abuse, it will need to be returned pre-paid. Parts and labor will be held to a minimum.

All costs will be quoted before any repairs are made.