



*No more
broken needles!*

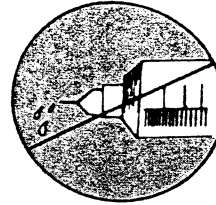

*No more
wasted vaccine!*

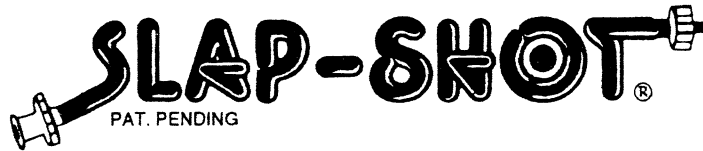

INSTRUCTIONS:

- Lock in needle to male of "Slap-Shot" and lock female end into syringe, filling both the syringe and the tube.
- To vaccinate, slap the needle into the animal at the proper vaccination sight and immediately squeeze the handle thus dispensing the vaccine.
- To mark animal hold marker in palm, upon injection simply mark.
- Pull back the syringe, which will quickly remove the needle from the animal.
- The needle and "Slap-Shot" remain intact and the animal undisturbed.
- **Reusable** - Simply wash with soapy water, rinse with clear water, and use over and over again.

Safety Glasses should be worn. Not responsible for accidental injection. (non-sterile)

Slap-Shot Co. Inc.
P.O.Box 972
McCook, NE 69001
Phone 308-345-2520
Fax 308-345-2884
Website: www.slAPSHOT-flex-vac.com

MADE IN THE U.S.A.

01-27-05



7 SLAP SHOT 30 INCH