


**SUPERVET™
DIGITAL THERMOMETER
WITH BEEPER**

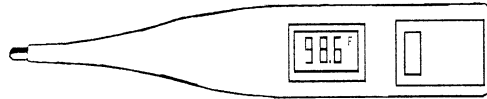

- Features 60 second readout
- Complete with battery and storage case
- Professional accuracy
- Beeps when peak temperature is reached

Instructions for use

1. Press ON/OFF button
2. Flashing "F" means Ready


General care and use

Clean your SUPERVET™ Thermometer by wiping it with soap and water or 70% Isopropyl Alcohol. The unit must never be submerged in any liquid or sterilized in any way (gas or steam autoclaves). Failure to follow these instructions will cause severe damage to your unit.

SYRVET, INC.
16200 Walnut St.
Waukee, Iowa
50263

800-727-5203
E-Mail address:
syrvet@worldnet.att.net
<http://www.syrvet.com>

Code: TA804

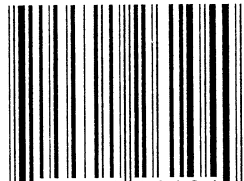

7 15501 26104 0