


Callicrate 'WEE' Bander™ Instructions

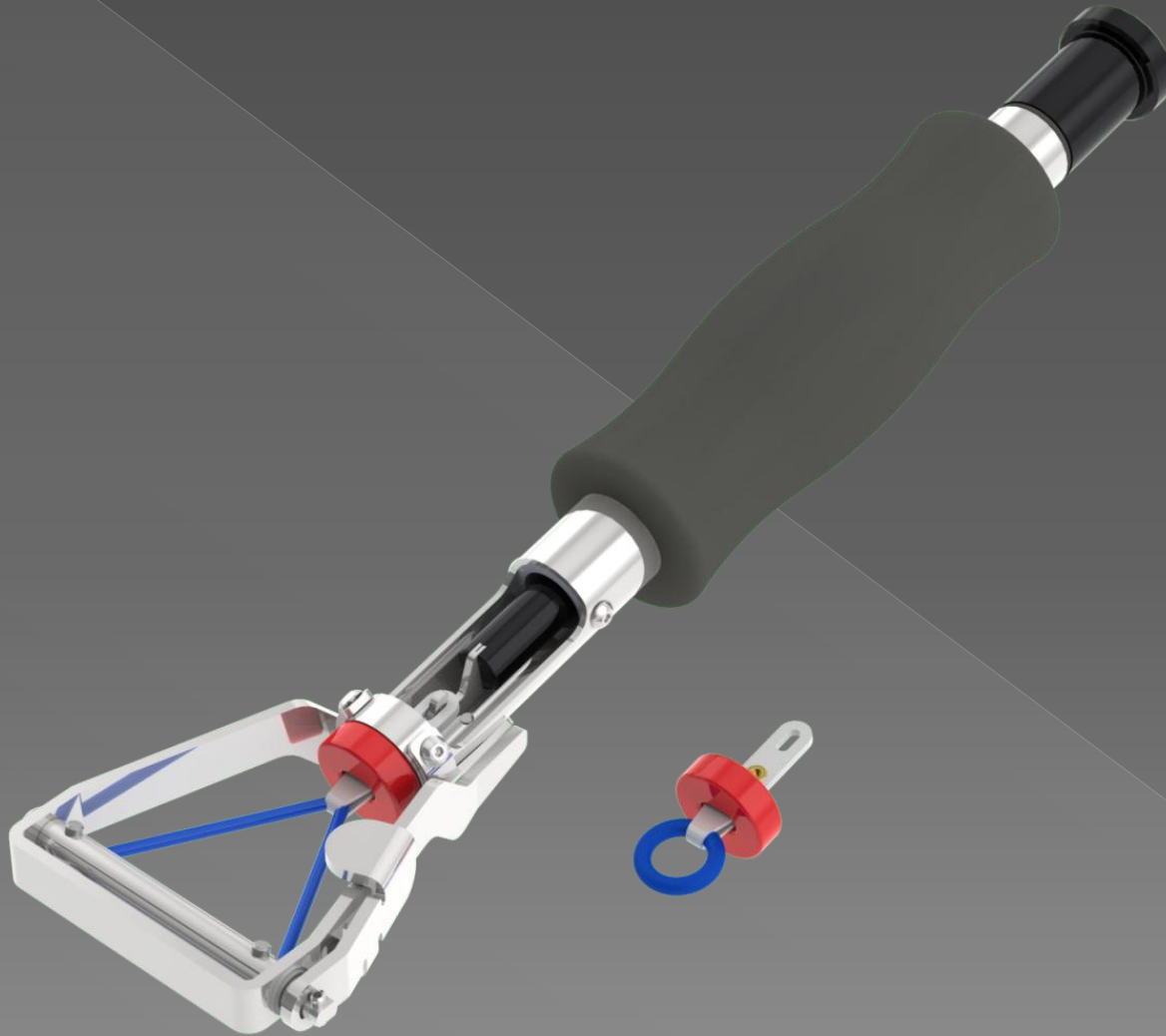

Step 1 – Load the Callicrate ‘WEE’ LoopTM



- At the handle end, rotate the tension rod counter clockwise from the locked position so the hook faces up and the handle slides in and out freely.
- Grasp the Callicrate ‘WEE’ BanderTM at the handle end with your hand cupped over the end of the tension rod.
- Insert Callicrate ‘WEE’ LoopTM into nose of bander and push the spring loaded tension rod forward and engage the metal hook with the loop.



Engage Hook

Step 2 – Expand the Callicrate ‘WEE’ Loop™

- Turn the ‘WEE’ Bander over using thumb to hold the loop on the bander nose and a finger to keep the release lever in place.
- Using the Callicrate Loop Stretcher or your fingers, stretch the ‘WEE’ Loop over the pegs as shown. The loop is now ready to apply above the testicles.

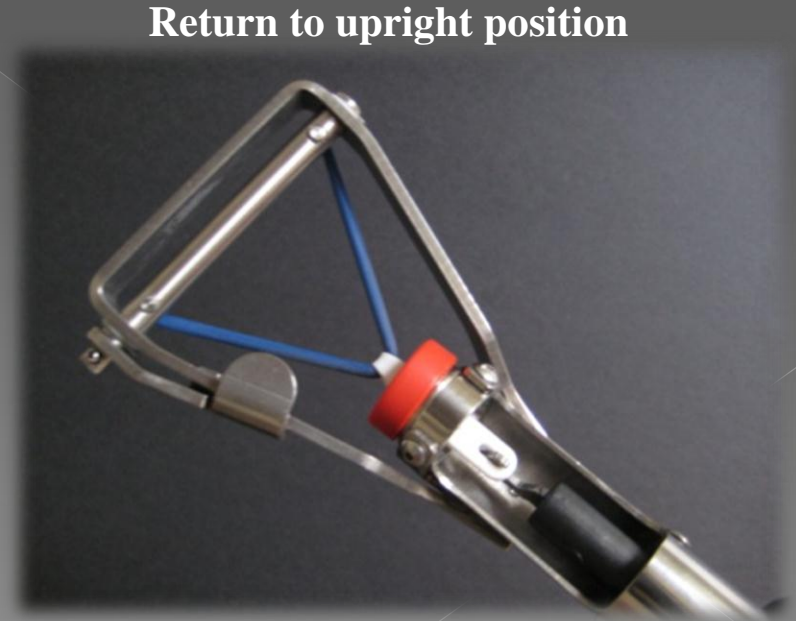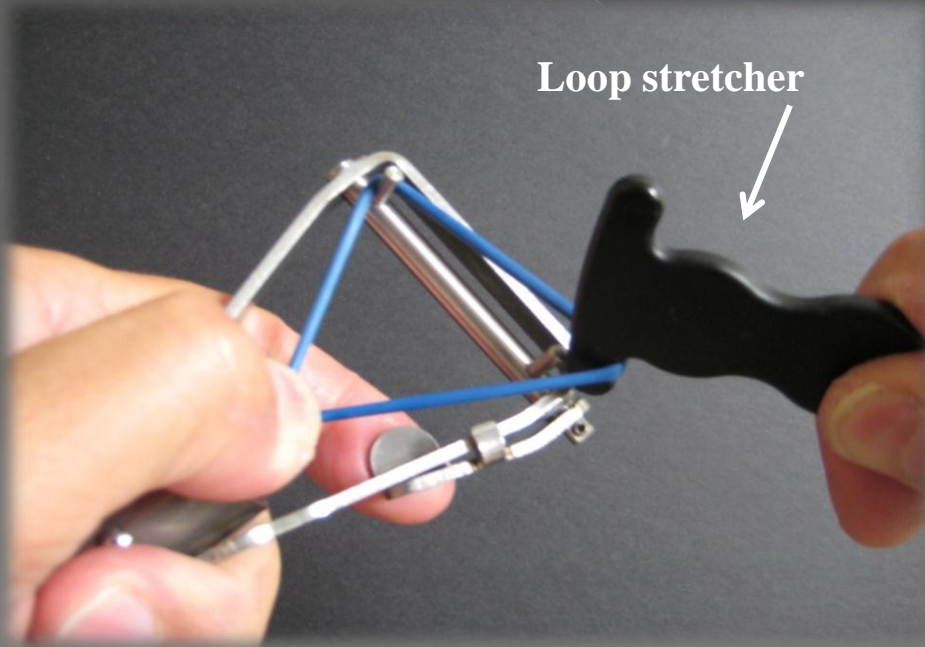

Step 3 – Place 'WEE' Loop on Animal

- With 'WEE' Bander right side up, slip BOTH testicles thru expanded loop opening.
- Push the loop release lever down and away from side bar to release the loop and capture testicles in the loop.

Caution: Always verify that both testicles are captured before releasing the loop.



Animal lying on side



Release lever



Step 4 – Apply Tension and Release

- Grasp the 'WEE' Bander handle firmly and draw the tension rod back until the **yellow** band is visible or until the rod hits the hard stop – proper tension is now achieved.
- Once tension is applied, let go of the tension rod. The tension rod will snap back and release the 'WEE' Loop from the hook, allowing the tool to be removed from the animal.



- Return tension rod to locked position.

Alternate Banding Method

- At **Step 3**, apply the loop with animal standing. You will be holding bander upside down.
- After releasing the loop around the testicles, firmly grasp the front of 'WEE' Bander with one hand and pull tension rod back with other hand.
- This method is dependent on feel to ensure **BOTH** testicles are captured in the loop. You may not be able to see the yellow band on the tension rod, so make sure the tension rod is pulled back until it hits the hard stop.



Animal Standing



Safety, Maintenance, & Registration

- Always wear protective eye wear when using the Callicrate 'WEE' Bander™.
- The tension handle can be easily removed for cleaning by pulling straight back and twisting 180 degrees.
- Clean with soapy water, rinse, and dry. Spray with silicone lubricant.
- When not using the Callicrate 'WEE' Bander™, push the tension handle all the way in and turn 90 degrees to lock in place.
- The Callicrate 'WEE' Bander™ is a precision instrument, treat it with care to ensure years of trouble free service.
- Warranty: Register your Callicrate 'WEE' Bander™ online @
www.CallicrateBanders.com

Loop Removal

If necessary to remove loop, snip as close to the plastic retainer as possible and gently pull plastic retainer off loop.

