



LEG WRAP STANDARD

THERAPEUTIC RELIEF FOR HORSES!

- Just soak in cold water...stays cool!
 - Reusable
- (Lasts two full days, then dry out and reuse)
- Moist heat therapy, too!

Outer Covering 100% Cotton

HOT & COLD THERAPY

LEG WRAP STANDARD

COOLER POLY CRYSTALS ABSORB & HOLD COLD WATER



To activate the Cooler Leg Wrap

Soak the Leg Wrap in cold water for 20 minutes. Advanced polymer cooling crystals encased within the wrap will plump up to provide a cold compress. During soaking period, occasionally squeeze the wrap to spread crystals evenly throughout the channels.

The Cooler Leg Wrap can also be used to provide moist heat therapy. Just soak briefly in hot water. Carefully test temperature before applying.

The Quick-Plump Method

The Cooler products plump up faster in hot water. To speed preparation time to as little as five minutes, set your cooler in hot water to plump it then transfer it to cool water for a few minutes.

Washing and Proper Care Instructions on Inside

E-Quest Inc.

205 ROUTE 541
MEDFORD, NJ 08055
1-800-235-3865

Model #41701 • Made in China



Washing Instructions

Completely dehydrate by hanging in open air before hand washing. Complete dehydration may take several days. Use only mild soap. Hand wash and rinse quickly. DO NOT machine wash or use harsh detergents.

Proper Care Instructions

Store by hanging in open air until completely dry. DO NOT store in a closed container or in plastic while damp as this will cause mildew. DO NOT freeze or spill salt water, gasoline, diesel fuel or alcohol on your Leg Wrap. DO NOT use with poultice. These will destroy the polymer & its effectiveness. If skin becomes irritated, wash with soap & water and discontinue use.

E-Quest Inc.