





Step 1 - Tools you will need are a Callicrate PRO Bander, Pro Bander Loops and Callicrate Cutter. You will also need SAFETY GLASSES (not Included with PRO Bander).

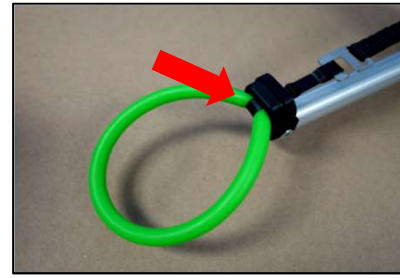

Step 2 – Install a PRO Bander Loop onto the PRO Bander nose and attach the pull strap hook to the loop pull tab. The loop clip should fit snugly on the nose of bander.



Step 3 – Using the spool ratchet, tension the pull strap just enough to snug the loop pull tab up to the loop clip.

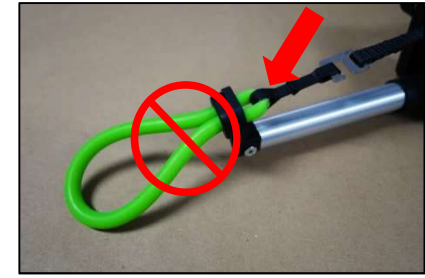

CAUTION! Do not pull the loop through the loop clip prior to installing loop on animal or clip will pull off the bander nose.



Step 4 – Hang the loop over your thumb and forth finger. Alternatively you can place thumb and 1<sup>st</sup> three fingers inside the loop (not shown above).




Step 5 – Straighten your fingers and flip your hand backward while pulling down on the bander.



Step 6 – From behind the bull, grasp the scrotum above the testicles and slide your hand down to force the testicles to the bottom of the scrotum.



Step 7 – Pull the bander down stretch the loop over the testicles.

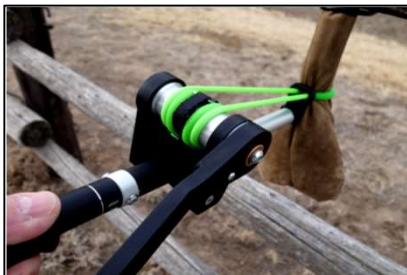

Step 8 – With the loop placed approximately 1 inch (25mm) above the testicles, tension the loop. Avoid placing loop to close to the belly.



Step 9 – Tension until the tension indicator moves all the way back to the scribed line.



Step 10 – Using the Callicrate Cutter, positioned as close to the tension spool as possible, cut the loop.



Step 11 – Visually verify loop is installed correctly. There should be two tails protruding from the clip approximately ½ inch (12mm).