

LidoBand™ XL

Solvat

Castration Bands- For castration of bulls and to deliver lidocaine as a local anesthetic. For use in bulls weighing between 250 to 750 pounds.

Description: LidoBand™ XL is a latex rubber band designed to be placed around the base of the scrotum of calves for castration. Each castration band is infused with 240mg of lidocaine, which provides local anesthesia. Lidocaine will quickly diffuse through the skin and remain in the local tissue due to vasoconstriction from the band.

Directions for use: Do not touch the band with bare fingers. Wear gloves when handling LidoBand XL. Because lidocaine is used in LidoBand XL, change gloves frequently when handling multiple bands. A tension band applicator (XL banders type) is recommended. Follow the manufacturer's directions for the tension band applicator when applying LidoBand XL. The properties of the latex changes with temperature. Very cold and very hot bands may break when expanded. Use bands at temperatures between 60°F and 85°F (15° and 30°C).

Precautions: Lidocaine is typically well tolerated. As with all local anesthetics, untoward effects may occur due to hypersensitivity.

LidoBand XL should not be used on animals with a known lidocaine hypersensitivity. Castration of cattle may result in the development of tetanus. For tetanus prevention, it is highly recommended to vaccinate with a tetanus toxoid at least three (3) weeks prior to castration. Consult your veterinarian on prevention of tetanus in cattle

User Safety: If skin comes in contact with LidoBand XL, wash affected area immediately with soap and water.

Storage: Keep stored in a cool, dry place. Do not leave bands in direct sunlight or allow exposure to high temperatures. Do not re-use the band.

For animal use only

Keep out of reach of children

California requires:

WARNING: Cancer Risk

www.P65Warnings.ca.gov