

A-Tag™


ONE-PIECE EAR TAG

Applicator


For application of Allflex A-Tag™ one-piece ear tags
A-Tag™ Application Instructions




Flip applicator pin out to load A-Tag™

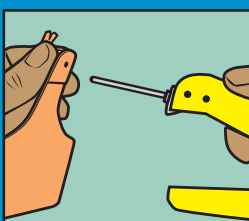


Fully insert applicator pin into A-Tag™ while rotating tag anchor

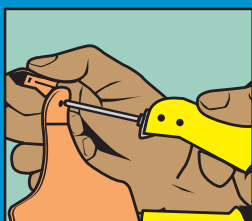


Seat tag anchor "v" onto pin so that pointed tip flexes outward

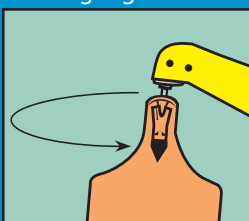

Rotate pin and A-Tag™ back into tagging position

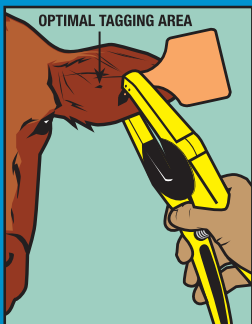

Position tag point between the two cartilage ribs in ear. Do not use existing ear tag holes

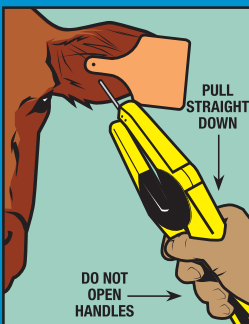

To apply tag, squeeze applicator handles together. Pull applicator straight down, away from animal



FRONT



BACK

Allflex®

ALLFLEX USA, INC.
P.O. Box 612266
Dallas/Fort Worth Airport, TX 75261-2266
(800) 989-TAGS (8247)
www.allflexusa.com

ALLFLEX CANADA
4135 Bérard
St-Hyacinthe, QC J2S 8Z8
866-505-TAGS