

CMT

JorVet™

California Mastitis Test CMT Liquid Concentrate

Dilute before use, will make one gallon.

To dilute concentrate:

1. Distilled water is preferred. Normal tap water can be used but should be tested for high alkaline pH. This is done by adding 1-2 drops of CMT to a tablespoon of water. Water can be used if color remains purple. Do not use acid water.
2. it is recommended to mix entire concentrate at one time. Pour concentrate into clean, empty gallon container. Add seven more pints of water for a full gallon.

To use:

Follow enclosed written directions

Contents: 16 fluid ounces

For invitro diagnostic use only.
Not for internal use.