

Oral Calf Feeder

Jorgensen Labs

The new JorVet Oral Calf Feeder/Drencher holds 2 ½ liters and can easily be operated by one person.

Product Concept:

This oral calf feeder is a collapsible 2 ½ liter bag. The bag is filled with electrolyte or nutritional solutions via the 3 tiered zipper. The slotted holes on the very top of the bag are convenient for hanging. The on-off clamp controls your flow. A rigid lexan tube with a round knob end makes proper placement in the esophagus much easier.

Directions For Use:

1. Fill container to desired level via ziplock opening.
2. Hang container via 3 slots at top of Bag.
3. The operator should face the bag in the same direction as calf.
4. Kneel on right side of animal and secure it with the left arm around the calf's neck.
5. Gently pass the rigid tube down the throat on the left side and continue into the esophagus.

CAUTION: The rigid tube must be in the esophagus, as administration of fluids into the trachea will result in death of the animal.

IMPORTANT: 4 to 5 inches of the solid tube should remain outside of the mouth.

6. Slowly release the on-off clamp. The container will empty in approximately 30 seconds. If the flow stops, gently reposition the tube.
7. Slowly remove the probe from the esophagus to avoid accidental aspiration after you are finished.
8. Wash and disinfect after every use. The Oral Calf Feeder should be completely dry before storage.

Made in China

Warning This product can expose you to chemicals including Diethylhexyl Phthalate (DEHP), which is known to the State of California to cause cancer or birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov