

Animalintex® Hoof Poultice Pad

3M™

Pre-cut hoof shape

- Aids in relief of equine foot injuries and conditions
- Aids in wound healing and relief of minor stiffness and soreness
- Convenient • No mess • Easy to apply and remove • Ready to use poultice in a pad – just add water

3 pads

Designed especially for the equine foot

Contains: Tragacanth 1.12g/dressing Boric Acid 0.39/dressing

How To Use:

Can be used as a hot or cold wet poultice, or as a dry wound dressing.

1. Gently clean application area.
2. Place poultice in clean, shallow container (plastic backing on top).
3. **For hot wet poultice**, pour hot, potable water (38°C/100°F) around edges of poultice in container.
4. **For cold wet poultice**, use cool, potable water.
5. When saturated, gently squeeze out excess water.
6. Position poultice (**plastic backing away from sole of foot**).
7. Cover poultice completely with 3M™ Vetrap™ Bandaging Tape.

Dry Wound Dressing

1. Gently clean application area.
2. Position poultice (**plastic backing away from sole**); cover poultice completely with 3M™ Vetrap™ Bandaging Tape.

Caution:

- **Change Poultice At Least Once Every 12 Hours**
- **For External Use only. For Animal Use Only.**
- Wash hands before and after use.
- Do not apply too hot.
- Apply poultice with the plastic backing away from animal.
- Do not bandage too tightly.
- If skin is broken, follow standard methods of wound care.
- **Indicated for use on horses and dogs. Not recommended for use on cats.**

Made in England