

Cap-Chur Charges

What Does A Cap-Chur Charge Do?

When a Syringe contacts the target animal, the Cap-Chur Charge is automatically triggered and the expanding gas generated drives the Rubber Plunger forward, injecting the animal with the liquid carried in the Syringe. **Note:** *You must use a Cap-Chur Charge each time you fire the Syringe, you Do Not need to use a Cap-Chur Charge for Target Practice.*

There are three strengths of Cap-Chur Charges:

50 Cap-Chur Charges

For 1cc Thru 3cc Syringes Part # 2361

Store in a cool, dry and safe place

KEEP OUT OF REACH OF CHILDREN

Manufactured in the USA by

Palmer Cap-Chur Equipment

421 Tidwell Rd, Powder Springs, Ga. 30127

770-942-4395

Cartridges for weapons, inert projectile-Class 1.4S

**1-3cc Cap-Chur Charges
(Yellow/Black)**

50 Cap-Chur Charges

For 4cc Thru 10cc Syringes Part # 2363

Store in a cool, dry and safe place

KEEP OUT OF REACH OF CHILDREN

Manufactured in the USA by

Palmer Cap-Chur Equipment

421 Tidwell Rd, Powder Springs, Ga. 30127

770-942-4395

Cartridges for weapons, inert projectile-Class 1.4S

**4-10cc Cap-Chur Charges
(Orange /Black)**

50 Cap-Chur Charges

For 15cc & 20cc Syringes Part # 2365

Store in a cool, dry and safe place

KEEP OUT OF REACH OF CHILDREN

Manufactured in the USA by

Palmer Cap-Chur Equipment

421 Tidwell Rd, Powder Springs, Ga. 30217

770-942-4395

Cartridges for weapons, inert projectile-Class 1.4S

**15cc Cap-Chur Charges
(White/Red)**



Note that one end of the Cap-Chur Charge is solid with rounded ends. The other end also has a rounded end but the center can be pushed down ('**Swaged**' end). The solid end must be put into the Rubber Plunger for the Syringe to work properly. **Note:** *Be careful not to use any object that will fit inside the charge. This could cause the firing pin to strike the primer.*