The goal when using the Cap-Chur projector is to have the syringe hit the animal rather softly (not hard like a bullet out of a gun). That lessens the likelihood of bounce-out and allows time for the liquid to be injected before the needle falls out of the animal.

Cap-Chur Disposable Aero Syringes—

- <u>Collared needles</u> work well for calves, and can be used on adult animals. They are a good choice for new producer-operators not used to the Cap-Chur equipment.
- <u>Grit Blasted needles</u> are rough on the surface, therefore fall out more readily than the collared, but not quite as readily as plain needles. They often make a good intermediary needle choice.
- <u>Plain needles</u> are often the preference for experienced Cap-Chur users.