

Equine Worm Test@Home

Perfect Pet

Equine Worm Test@Home Directions:

- 1) Return the sample **and the clamshell** or we will not process. Include the entire original 3 part lab form (no copies).
- 2) **PLEASE PRINT ON THE LAB FORM OR TYPE ON A SEPARATE SHEET** so we can read the info, **especially your email and phone!!!**
- 3) Label the 3X5" zip-lock bags with your name and animal's name.
- 4) Secure 2 Tbsp. of feces (**NO MORE THAN THAT**) and place inside both zip-lock bags so it does not leak and tape each zip-lock bag closed.
- 5) If the sample is watery; wrap all the bags in 2 layers of paper towel to absorb any leakage. **You must package the sample so there is no leakage or odor—OR ELSE** the post office, etc. can refuse to process the sample. **YOU MUST USE THE CLAMSHELL.**
- 6) **Place the sample and the printed/typed lab form inside the plastic clamshell.**
- 7) **Place the clamshell inside the white envelope—PLEASE.**
- 8) **Add seven stamps and mail First Class Mail.** If possible mail on Monday.
- 9) You can mail as USPS Priority, FedEx, or UPS, but check our website (shipping tab) for the correct address. **There will be no refunds if you do not follow all the directions and a \$10 fee to resend via email your lab form if we can't read your form.** Most samples sent without using the clamshell are destroyed by the post office machines.
- 10) If you observe live worms, separate them from the stool, wash so only the worm is visible, and take a photo. Include the photo or email the photo to info@perfectpetproducts.com and include the worm(s) in a separate bag. Please note on the lab form that worms are included.
- 11) Request the **McMaster's, Lung Worm Test, and Fecal Occult Blood.**
- 12) You may **send multiple white envelopes inside a larger mailer for faster results.** You can mail by USPS Priority or Express Mail., FedEx Express, UPS Express, etc. Check our website (shipping) for the correct address.

YOU MUST SEND THE SAMPLE INSIDE THE PLASTIC CLAMSHELL!