

# **Tubbease® Hoof Sock**

## ***Shires Equestrian***

**Making hoof tubbing easy**

**Original Tubbease® Hoof Sock** designed and developed in New Zealand

### **Fitting Instructions**

1. After preparing or medicating the hoof, open sock wide and pull Tubbease over hoof.
2. Check Tubbease is fitting hoof correctly and then pull sock up firmly.
3. With sock taut, fit hook and loop strap to lower pastern with hoof side facing sock.  
Ensure one finger space under strap.
4. With hoof on the ground, simply pour in treatment solution as required, or submerge fitted hoof into a bucket of your chosen solution for deeper soaking.