

# Hoof Hug™ Tape

## America's Acres

- **Never Use Duct Tape Again for Wrapping a Hoof**
- **Easily Conforms to Hoof—Pre-Cut Folds—Hoof Shaped**
- **Non-Slip Surface for Superior Ground Contact**

**Applications:** Sole Abscesses, Sores or Bruises, Medicating a Hoof, Thrush, Thrown Shoe, Hoof & Heel Cracks

**Caring For Hoof Issues Made easy**

**No Need to Duct Tape Your Horse's Hoof Again**

**HOOF HUG™ TAPE with Non-Slip Surface for Horses With Shoes or Barefoot**

**READ THE INSTRUCTIONS COMPLETELY BEFORE APPLYING**

**One size fits most. For Custom fitting Extend Tab Cuts, and Trim as Needed**

- 1--Must have Clean & Dry Hoof & Sole Area
- 2--Use hoof packing on sole area as needed
- 3--Peel paper from back straight edge of Hoof Hug™ Tape 2 inches
- 4--Securely Stick the exposed 2 inches of tape to the rear hoof heal area of the hoof
- 5--Peel All backing paper off Tape pulling from heel to toe until hoof sole/shoe is covered & adhered
- 7--Next, press the 2 Side Tape Tabs to both side walls of hoof firmly
- 8--Press the 3 Toe Tab from your left to right against hoof toe area overlapping each Tab
- 9--Press firmly on all sides of hoof for a uniform tight fit
- 10--DO NOT place Tape on sensitive skin area above hoof walls.

**Note** *Horses with Large Hooves or turned out may need 2<sup>nd</sup> Hoof Hug added for durability*

America's Acres Inc. restrict the sale or resale of this in the State of California

Made in USA