

# Vetericyn Plus® Antimicrobial Dressing Gel

## Vetericyn®

### Thick Barrier Wound Care

- Use under dressings & bandages
- Cuts & abrasions
- Sores & irritations

**Vetericyn Plus® Antimicrobial Dressing Gel adheres to the site of application, creating a wound protecting barrier.**

- For use under dressings & bandages
- Helps coat, moisturize & protect wounds, scrapes, abrasions & minor wounds
- Aids wound protection to help reduce the chance of potential infection
- Safe for all animal species at all life-stages

Veterinarian recommended

Safe if licked or ingested

No antibiotics or steroids

**Indications:** Vetericyn Plus® Antimicrobial Dressing Gel is intended for the management of wounds including skin abrasions, lacerations, irritations, cuts, burns and intact skin; providing a proper moisture balance throughout the phases of wound healing. Relieves itch and pain associated with skin irritations. Optimal to use with dressings and bandages. Safe for all animal species at all life stages.

**Directions for use:** Shake well before use. Move excess hair from wound area. Clip if necessary. Open cap and invert bottle. Squeeze out gel and saturate the affected area and cover with a bandage, or wrap with dressing. For best results, flush and clean with Vetericyn Wound Care Spray if dirt or debris exist and use in conjunction with Vetericyn Plus Antimicrobial Dressing Gel for proper protection and moisture balance during healing stages. Repeat at each dressing or bandage change 3-4 times per day or as necessary. Faster healing is achieved when wounds are cleaned and maintained moist. No rinsing necessary. Safe for use around mouth, nose, ears and eyes.

**Use Precautions:** External use only, not for injection. Animal use only, not for humans. Keep out of reach of children. Discontinue use if irritation or systemic condition develops and contact your veterinarian. Will not stain and contains no harsh chemicals or dyes.

**Ingredients: Active:** Hypochlorous Acid (0.012%)

**Inactive:** Electrolyzed Water, Sodium Chloride, Sodium Hypochlorite, Lithium Magnesium Sodium Silicate, Sodium Bicarbonate, Phosphates, Sodium Sulfate

**Storage & Disposal:** Store at room temperature. Do not allow to freeze. Do not use after expiration date. After use, rinse nozzle with warm water to reduce chance of gel drying at the tip. Close cap. No special disposal requirements.

Innovacyn products are manufactured in a FDA QSR compliant facility. Vetericyn Plus contains Hypochlorous Acid, a known antimicrobial, which serves as a preservative to inhibit growth of microorganisms in the product.

Made in the USA