

LidoBand™

Alberta Veterinary Laboratories/Solvat Ltd.

Castration Bands-For castration of calves and lambs, and to deliver lidocaine as a local anesthetic. For use in calves under 250 pounds or lambs under 50 pounds.

Description: LidoBand™ is a latex rubber band designed to be placed around the base of the scrotum of calves and lambs for castration. Each castration band is infused with 80mg of lidocaine, which provides local anesthesia beginning at 2 hours post-application and lasting up to 42 days.

Directions for use: Each band is individually packaged in a foil-sealed blister. Do not touch the band with bare fingers. Wear gloves when applying LidoBAnd. Only apply LidoBAnd with a commercial band applicator (not included).

Step 1 Puncture the blister surface with the band applicator.

Step 2 Remove the band from packaging while loaded on the prongs of the applicator.

Step 3 Squeeze the applicator handle to spread the band.

Step 4 Pull the testicles into the scrotum, place the band above the testicles and remove the applicator.

Precautions: Lidocaine is typically well tolerated. As with all local anesthetics, untoward effects may occur due to hypersensitivity. LidoBand should not be used on animals with a known lidocaine hypersensitivity.

User Safety: if skin comes into contact with LidoBand, wash affected area immediately with soap and water.

Storage: Keep stored in a cool, dry place. Do not re-use the band.

California requires:

WARNING: Cancer Risk
[www.P65 Warnings.ca.gov](http://www.P65Warnings.ca.gov)