

# Pregnostyx On-Site Pregnancy Tests

## *Biotracking*

**CATTLE All Breeds**

**28+ Days Post Breeding**

<b>FAST</b>	<b>EASY</b>	<b>PROVEN</b>
Results in <b>30 minutes</b>	Ready for <b>on-farm use</b>	Trusted technology

**6 Test Kits**

### **CONTENTS**

- 6 Pregnostyx Tests
- 6 Vacuum Tubes
- 6 Pipettes
- 6 Chase Capsules
- 6 Needles
- 2 Reusable Needle Holders

### **INSTRUCTIONS**

- 1 Draw Blood Sample** from the cattle test subject
- 2 Prepare Test** Lay test flat on a table
- 3 Transfer Sample**
- 4 Place Sample** Squeeze pipette into sample port
- 5 Add Chase** Squeeze 3 drops into sample port
- 6 Results After 30 Minutes**