




## Easyboot & EasyShoe Size Charts

### How to Measure

The biggest determining factor in EasyCare hoof boot\* and EasyShoe success is fit. Always measure as soon as possible after a fresh trim. Waiting to take measurements can result in a poor fit.

Check the product size chart before starting your measurements. If the sizes are shown in millimeters only, be sure to measure in millimeters. Measuring in inches and converting to millimeters will render the measurement process inaccurate.

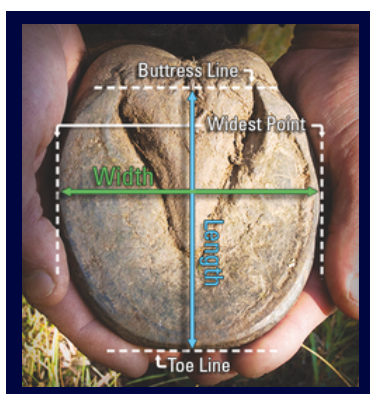

1. Measure the width of the hoof across the bottom at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurements with the appropriate size chart. Please note: each model has its own unique size chart.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are more than one size apart, your selected model is not recommended. \*\* Check your measurements against an alternative product in the EasyCare line - another model may accommodate your horse's hoof shape better.

\* Hoof boots should be snug before tightening. After tightening, the boot should be a close fit around the hoof wall. You should not be able to turn or twist the boot.

\*\* If the hoof measurements are more than one size apart in length and width, please call an EasyCare Product Specialist at 800.447.8836.

### Chart Reference

PERFORMANCE	
Easyboot Epic	1
Easyboot Flip Flop	2
Easyboot Love Child (LC)	3
Easyboot Glove Soft	4
Easyboot Glue-On	4
PLEASURE	
Easyboot Back Country	4
Easyboot Mini	7
Old Mac's G2	8
Easyboot Sneaker	9
Easyboot Trail & Trail Original	8
THERAPY	
Easyboot Remedy	10
Easyboot Remedy Ultimate	10
Easyboot Cloud	11
Easyboot Rx	12
Easyboot Rx2	13
Easyboot Zip	14
EASYSHOE	
EasyShoe Compete	15
EasyShoe Flex Heart Bar	16
EasyShoe Performance & N/G	17
EasyShoe Sport	18
EasyShoe 3D	19
EasyShoe Speed	20
EasyShoe Race	21
EasyShoe Roller	22
EasyShoe Adapter	23
EasyShoe One Glue	24
EasyShoe One Nail	25
EasyShoe Versa Grip, Gold, Light	26
EasyShoe Versa Grip Glue, Octo	27
EasyShoe Versa Grip Wedge	28
EasyShoe Versa Grip Rocker	29

**CHART 1: Easyboot Epic**

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>Pony</b>	3 3/16 - 3 3/8	3 3/8 - 3 5/8	78-85	85-92
<b>000</b>	3 3/8 - 3 13/16	3 5/8 - 4 3/16	86-97	93-105
<b>00</b>	3 13/16 - 4 3/8	4 3/16 - 4 5/8	98-110	106-117
<b>0</b>	4 3/8 - 4 9/16	4 5/8 - 4 15/16	111-116	118-124
<b>1</b>	4 9/16 - 4 7/8	4 15/16 - 5 1/8	117-123	125-130
<b>2</b>	4 7/8 - 5 1/8	5 1/8 - 5 9/16	124-130	131-140
<b>3</b>	5 1/8 - 5 1/2	5 9/16 - 5 15/16	131-139	141-150
<b>4</b>	5 1/2 - 5 15/16	5 15/16 - 6 7/16	140-150	151-163
<b>5</b>	5 15/16 - 6 7/16	6 7/16 - 6 7/8	151-163	164-174
<b>6</b>	6 7/16 - 6 7/8	6 7/8 - 7 7/16	164-175	175-188
<b>7</b>	6 7/8 - 7 9/16	7 7/16 - 7 3/4	176-192	189-195

**CHART 2: Easyboot Flip Flop**

Size	Width (mm)	Length (mm)
<b>0</b>	104-107	Trim to size
<b>0.5</b>	108-111	Trim to size
<b>1</b>	112-115	Trim to size
<b>1.5</b>	116-119	Trim to size
<b>2</b>	120-123	Trim to size
<b>2.5</b>	124-127	Trim to size
<b>3</b>	128-131	Trim to size

**CHART 3: Easyboot Love Child (LC)**

Size	Width (mm)	Length (mm)
<b>0</b>	104-107	111-118
<b>0.5</b>	108-111	116-123
<b>1</b>	112-115	121-128
<b>1.5</b>	116-119	126-133
<b>2</b>	120-123	132-139
<b>2.5</b>	124-127	134-141
<b>3</b>	128-131	139-146
<b>3.5</b>	132-135	142-149
<b>4</b>	136-139	145-152

**CHART 4: Easyboot Glove Soft, Back Country, Glue-On**

Regular Sizes		
Size	Width (mm)	Length (mm)
<b>00</b>	96-99	106-109
<b>00.5</b>	100-103	110-113
<b>0</b>	104-107	114-117
<b>0.5</b>	108-111	118-121
<b>1</b>	112-115	122-125
<b>1.5</b>	116-119	126-129
<b>2</b>	120-123	130-133
<b>2.5</b>	124-127	134-137
<b>3</b>	128-131	138-141
<b>3.5</b>	132-135	142-145
<b>4</b>	136-139	146-149
<b>4.5</b>	140-143	150-153
<b>5</b>	144-147	154-157
Wide Sizes		
Size	Width (mm)	Length (mm)
<b>0W</b>	114-117	114-117
<b>0.5W</b>	118-121	118-121
<b>1W</b>	122-125	122-125
<b>1.5W</b>	126-129	126-129
<b>2W</b>	130-133	130-133
<b>2.5W</b>	134-137	134-137
<b>3W</b>	138-141	138-141

*Always measure in millimeters as directed in our "How to Measure" guide.*

*A Fit Kit is available to ensure correct sizing.*

**CHART 7: Easyboot Mini**

Size	Width (mm)	Length (mm)
<b>Mini</b>	44-49	49-54
<b>Mini 1</b>	50-55	55-64
<b>Mini 2</b>	56-65	65-72
<b>Mini 3</b>	66-76	73-86

**CHART 8: Old Mac's G2, Easyboot Trail & Trail Original**

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>0</b>	3 15/16 - 4 1/8	4 1/8 - 4 15/16	100-105	105-110
<b>1</b>	4 1/8 - 4 5/16	4 5/16 - 4 1/2	105-110	110-115
<b>2</b>	4 5/16 - 4 1/2	4 1/2 - 4 3/4	110-115	115-120
<b>3</b>	4 1/2 - 4 3/4	4 3/4 - 4 15/16	115-120	120-125
<b>4</b>	4 3/4 - 4 15/16	4 15/16 - 5 1/8	120-125	125-130
<b>5</b>	4 15/16 - 5 1/8	5 1/8 - 5 5/16	125-130	130-135
<b>6</b>	5 1/8 - 5 5/16	5 5/16 - 5 1/2	130-135	135-140
<b>7</b>	5 5/16 - 5 1/2	5 1/2 - 5 11/16	135-140	140-145
<b>8</b>	5 1/2 - 5 11/16	5 11/16 - 5 15/16	140-145	145-150
<b>9</b>	5 11/16 - 5 15/16	5 15/16 - 6 1/8	145-150	150-155
<b>10</b>	5 15/16 - 6 1/8	6 1/8 - 6 5/16	150-155	155-160
<b>11</b>	6 1/8 - 6 1/2	6 5/16 - 6 11/16	155-165	160-170
<b>12</b>	6 1/2 - 6 7/8	6 11/16 - 7 1/8	165-175	170-180

**CHART 9: Easyboot Sneaker**

Regular Sizes		
Size	Width (mm)	Length (mm)
<b>00</b>	90-100	95-105
<b>0</b>	100-110	105-115
<b>1</b>	110-120	115-125
<b>2</b>	120-130	125-135
<b>3</b>	130-140	135-145
<b>4</b>	140-150	145-155
<b>5</b>	150-160	155-165
Narrow Sizes		
Size	Width (mm)	Length (mm)
<b>00</b>	80-90	95-105
<b>0</b>	90-100	105-115
<b>1</b>	100-110	115-125
<b>2</b>	110-120	125-135
<b>3</b>	120-130	135-145
<b>4</b>	130-140	145-155
<b>5</b>	140-150	155-165

**CHART 10: Easyboot Remedy & Remedy Ultimate**

Size	Width (in)	Width (mm)
<b>Medium</b>	Up to 4 3/4"	Up to 120.5
<b>Large</b>	Up to 6 1/2"	Up to 160

**CHART 11: Easyboot Cloud**

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>00</b>	3 1/2 - 3 7/8	4 - 4 3/8	89-99	102-112
<b>0</b>	3 7/8 - 4 1/4	4 3/8 - 4 5/8	99-108	112-118
<b>1</b>	4 1/4 - 4 7/8	4 5/8 - 5 1/8	108-124	118-131
<b>2</b>	4 7/8 - 5 1/4	5 1/8 - 5 3/8	124-133	131-137
<b>3</b>	5 1/4 - 5 7/8	5 3/8 - 5 7/8	133-149	137-150
<b>4</b>	5 7/8 - 6 1/4	5 7/8 - 6 1/4	149-159	150-159
<b>5</b>	6 1/4 - 6 11/16	6 1/4 - 6 11/16	159-170	159-170
<b>6</b>	6 11/16 - 6 15/16	6 11/16 - 6 15/16	170-177	170-177
<b>7</b>	6 15/16 - 7 3/8	6 15/16 - 7 3/8	177-188	177-188
<b>8</b>	7 3/8 - 7 1/2	7 3/8 - 7 7/8	188-191	188-201

**CHART 12: Easyboot Rx**

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>00</b>	3 13/16 - 4 3/8	4 3/16 - 4 5/8	98-110	106-117
<b>0</b>	4 3/8 - 4 9/16	4 5/8 - 4 15/16	111-116	118-124
<b>1</b>	4 9/16 - 4 7/8	4 15/16 - 5 1/8	117-123	125-130
<b>2</b>	4 7/8 - 5 1/8	5 1/8 - 5 9/16	124-130	131-140
<b>3</b>	5 1/8 - 5 1/2	5 9/16 - 5 15/16	131-139	141-150
<b>4</b>	5 1/2 - 5 15/16	5 15/16 - 6 7/16	140-150	151-163
<b>5</b>	5 15/16 - 6 7/16	6 7/16 - 6 7/8	151-163	164-174
<b>6</b>	6 7/16 - 6 7/8	6 7/8 - 7 7/16	164-175	175-188
<b>7</b>	6 7/8 - 7 9/16	7 7/16 - 7 3/4	176-192	189-195

**CHART 13: Easyboot Rx2**

Size	Width (mm)	Length (mm)
<b>000</b>	70-80	80-90
<b>00</b>	80-90	90-100
<b>0</b>	90-100	100-110
<b>1</b>	100-110	110-120
<b>2</b>	110-120	120-130
<b>3</b>	120-130	130-140
<b>4</b>	130-140	140-150
<b>5</b>	140-150	150-160
<b>6</b>	150-160	160-170
<b>7</b>	160-170	170-180

**CHART 14: Easyboot Zip**

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>0</b>	3 - 3 13/16	3 - 3 13/16	77-97	77-97
<b>1</b>	3 7/8 - 4 11/16	3 7/8 - 4 11/16	98-119	98-119
<b>2</b>	4 3/4 - 5 5/16	4 3/4 - 5 5/16	120-136	120-136
<b>3</b>	5 3/8 - 6 3/8	5 3/8 - 6 3/8	137-162	137-162
<b>4</b>	6 7/16 - 7 1/16	6 7/16 - 7 1/16	163-180	163-180

**CHART 15: EasyShoe Compete**

Size	Width (mm)	Length (mm)
<b>0</b>	95-118	111-118
<b>1</b>	105-125	119-127
<b>2</b>	115-135	128-136
<b>3</b>	125-145	137-143

**CHART 16: EasyShoe Flex Heart Bar**

Size	Width (mm)	Length (mm)
<b>0</b>	100-113	127
<b>1</b>	106-121	135
<b>2</b>	110-129	144
<b>3</b>	116-137	153
<b>4</b>	122-143	160
<b>5</b>	124-148	168
<b>6</b>	134-158	175

**CHART 17: EasyShoe Performance & Performance N/G**

Size	Width (mm)	Length (mm)
<b>0</b>	105-111	108-115
<b>1</b>	112-119	116-123
<b>2</b>	120-127	124-131
<b>3</b>	128-135	132-140
<b>4</b>	136-143	141-149
<b>5</b>	144-152	150-159

**CHART 18: EasyShoe Sport**

Size	Width (mm)	Length (mm)
<b>0</b>	100-110	114-117
<b>0.5</b>	105-115	118-121
<b>1</b>	110-120	122-125
<b>1.5</b>	115-125	126-129
<b>2</b>	120-130	130-133
<b>2.5</b>	125-135	134-137
<b>3</b>	130-140	138-141

**CHART 19: EasyShoe 3D**

Front Sizes		
Size	Width (mm)	Length (mm)
<b>118F</b>	112-118	128
<b>122F</b>	116-122	132
<b>126F</b>	120-126	136
<b>130F</b>	124-130	140
<b>134F</b>	128-134	145
Hind Sizes		
Size	Width (mm)	Length (mm)
<b>110H</b>	104-110	124
<b>114H</b>	108-114	128
<b>118H</b>	112-118	133
<b>122H</b>	116-122	137
<b>126H</b>	120-126	142

**CHART 20: EasyShoe Speed, Speed Metal, & Speed Metal Trac**

Front Sizes		
Size	Width (mm)	Length (mm)
<b>114F</b>	108-114	123
<b>118F</b>	112-118	128
<b>122F</b>	116-122	132
<b>126F</b>	120-126	136
<b>130F</b>	124-130	140
<b>134F</b>	128-134	145
<b>138F</b>	132-138	150
<b>142F</b>	136-142	154
<b>146F</b>	140-146	158
Hind Sizes		
Size	Width (mm)	Length (mm)
<b>106H</b>	100-106	119
<b>110H</b>	104-110	124
<b>114H</b>	108-114	128
<b>118H</b>	112-118	133
<b>122H</b>	116-122	137
<b>126H</b>	120-126	142
<b>130H</b>	124-130	146
<b>134H</b>	128-134	151

### CHART 21: EasyShoe Race

EasyShoe Race Wedge sizes correspond to shoe size.

Size	Width (mm)	Length (mm)
3	114-118	125
4	118-122	128
5	122-126	130
6	126-131	136
7	131-136	142
8	136-142	148

### CHART 22: EasyShoe Roller

EasyShoe Roller Wedge sizes correspond to shoe size.

Size	Width (mm)	Length (mm)
3	98-104	110
4	104-110	115
5	110-115	120
6	115-121	127
7	121-127	135
8	127-133	140
9	133-138	145
10	138-145	150
11	145-152	158

### CHART 23: EasyShoe Adapter

Accessories correspond to shoe size  
(Wedge, Clog, Roller, and Tenotomy.)

Size	Width (mm)	Length (mm)
118F	112-118	128
122F	116-122	132
126F	120-126	136
130F	124-130	140
134F	128-134	145

### CHART 24: EasyShoe One Glue

EasyShoe One Glue is elongated to fit both fronts & hinds.  
Use rasp or grinder to adjust length.

Front Sizes		
Size	Width (mm)	Length (mm)
114F	108-114	133
118F	112-118	137
122F	116-122	141
126F	120-126	146
130F	124-130	151
134F	128-134	155
138F	132-138	159

### CHART 25: EasyShoe One Nail

Front Sizes		
Size	Width (mm)	Length (mm)
118F	112-118	131
122F	116-122	135
126F	120-126	139
130F	124-130	143
134F	128-134	148
138F	132-138	153
Hind Sizes		
Size	Width (mm)	Length (mm)
114H	108-114	131
118H	112-118	136
122H	116-122	140
126H	120-126	145
130H	124-130	149
134H	128-134	154

**CHART 26:** EasyShoe Versa Grip & Versa Grip Light (sizes 102-206)  
Versa Grip Gold (sizes 102-170 only)

Front Sizes			
Size	Width (mm)	Length (mm)	Thickness (mm)
<b>102F</b>	96-102	113	16
<b>106F</b>	100-106	117	16
<b>110F</b>	104-110	122	16
<b>114F</b>	108-114	126	16.2
<b>118F</b>	112-118	131	16.2
<b>122F</b>	116-122	135	16.2
<b>126F</b>	120-126	139	16.2
<b>130F</b>	124-130	143	16.2
<b>134F</b>	128-134	148	16.2
<b>138F</b>	132-138	153	16.7
<b>142F</b>	136-142	157	16.7
<b>146F</b>	140-146	161	16.7
<b>150F</b>	144-150	166	17.2
<b>154F</b>	148-154	170	17.2
<b>158F</b>	152-158	175	17.2
<b>162F</b>	156-162	179	17.8
<b>166F</b>	160-166	183	17.8
<b>170F</b>	164-170	188	17.8
<b>174F</b>	168-174	192	
<b>178F</b>	172-178	196	
<b>182F</b>	176-182	200	
<b>186F</b>	180-186	205	
<b>190F</b>	184-190	209	
<b>194F</b>	188-194	214	
<b>198F</b>	192-198	218	
<b>202F</b>	196-202	222	
<b>206F</b>	200-206	227	

Hind Sizes			
Size	Width (mm)	Length (mm)	Thickness (mm)
<b>102H</b>	96-102	118	16
<b>106H</b>	100-106	122	16
<b>110H</b>	104-110	127	16
<b>114H</b>	108-114	131	16.2
<b>118H</b>	112-118	136	16.2
<b>122H</b>	116-122	140	16.2
<b>126H</b>	120-126	145	16.2
<b>130H</b>	124-130	149	16.2
<b>134H</b>	128-134	154	16.2
<b>138H</b>	132-138	159	16.7
<b>142H</b>	136-142	163	16.7
<b>146H</b>	140-146	168	16.7
<b>150H</b>	144-150	173	17.2
<b>154H</b>	148-154	177	17.2
<b>158H</b>	152-158	182	17.2
<b>162H</b>	156-162	186	17.8
<b>166H</b>	160-166	191	17.8
<b>170H</b>	164-170	195	17.8
<b>174H</b>	168-174	200	
<b>178H</b>	172-178	205	
<b>182H</b>	176-182	209	
<b>186H</b>	180-186	213	
<b>190H</b>	184-190	218	
<b>194H</b>	188-194	222	
<b>198H</b>	192-198	227	
<b>202H</b>	196-202	232	

**CHART 27: EasyShoe Versa Grip Glue**  
(sizes 74F-150F, 90H-150H) &  
EasyShoe Versa Grip Octo (sizes 102-150 only)

Front Sizes		
Size	Width (mm)	Length (mm)
74F	70-74	81
78F	74-78	85
82F	78-82	90
86F	82-86	95
90F	86-90	98
94F	90-94	102
98F	94-98	106
102F	96-102	110
106F	100-106	114
110F	104-110	119
114F	108-114	123
118F	112-118	128
122F	116-122	132
126F	120-126	136
130F	124-130	140
134F	128-134	145
138F	132-138	150
142F	136-142	154
146F	140-146	158
150F	144-150	163
Hind Sizes		
Size	Width (mm)	Length (mm)
90H	86-90	102
94H	90-94	106
98H	94-98	110
102H	96-102	115
106H	100-106	119
110H	104-110	124
114H	108-114	128
118H	112-118	133
122H	116-122	137
126H	120-126	142
130H	124-130	146
134H	128-134	151
138H	132-138	156
142H	136-142	160
146H	140-146	165
150H	144-150	170

**CHART 28: EasyShoe Versa Grip Wedge**

Front Sizes		
Size	Width (mm)	Length (mm)
114F	108-114	126
118F	112-118	131
122F	116-122	135
126F	120-126	139
130F	124-130	143
134F	128-134	148
138F	132-138	153
142F	136-142	157
146F	140-146	161
150F	144-150	166
Hind Sizes		
Size	Width (mm)	Length (mm)
114H	108-114	131
118H	112-118	136
122H	116-122	140
126H	120-126	145
130H	124-130	149
134H	128-134	154
138H	132-138	159
142H	136-142	163
146H	140-146	168
150H	144-150	173

**CHART 29: EasyShoe Versa Grip Rocker**

Front Sizes		
Size	Width (mm)	Length (mm)
114F	108-114	126
118F	112-118	131
122F	116-122	135
126F	120-126	139
130F	124-130	143
134F	128-134	148
138F	132-138	153
142F	136-142	157
146F	140-146	161
150F	144-150	166