

# **Restore by Noxano®**

## **Noxano® Animal Health**

### **5 x 7 cm Veterinary Wound Pad**

**Restore by Noxano Restore** helps veterinarians speed or restart wound healing after injury, surgery, or disease, to rapidly return patients to full health. Rapid healing reduces risk of infection, number of dressing changes, and clinic workload. This decreases stress on the animal and owner while lowering treatment cost.

**Indications:** Restore is recommended for infection control and treatment of all wounds in small and large animals, e.g., surgical incisions, skin grafts and flaps, degloving injuries, lacerations, bites, and minor scrapes and cuts. Restore can be used with existing wound technology such as skin grafts, negative pressure, hyperbaric oxygen, laser, etc., if desired.

1. Open package.
2. Add 15 mL water, reseal & lay flat. Wait 1 min.
3. Apply pad to wound and wrap to hold in place.\*

Activate **Restore by Noxano** prior to use – do NOT cut.

\*Apply the activated pad directly to the wound. Use a secondary dressing to hold Restore in place. Optionally, an occlusive dressing can be used for extended lifetime.

### **For Veterinary Use Only.**

For more information call 1-513-202-6052 or email [contact@noxano.com](mailto:contact@noxano.com).  
Video instructions available at [www.noxsano.com](http://www.noxsano.com).

### **Box Contents:**

5 individually packed 5 x 7 cm pads

- Do Not Reuse
- Store at Room Temperature 10°C-25°C (50°F-77°)
- Store Dry
- Do Not Use if Damaged
- See Instructions for Use